

# BF600

---

## Downdraft Work Cell



## REFERENCE MANUAL

Rev 6-12-14©



21750 Coolidge Highway • Oak Park, MI 48237 USA  
+1 800-362-8491 • 248-291-2040 • Email: [info@mopec.com](mailto:info@mopec.com)

## **IMPORTANT INFORMATION**

**MODEL NO.: BF600**

**SERIAL NO.:**

**DATE OF PURCHASE:**

**DATE OF INSTALLATION:**

**INSTALLATION NOTES:**

---

---

---


---

---

---

---

---

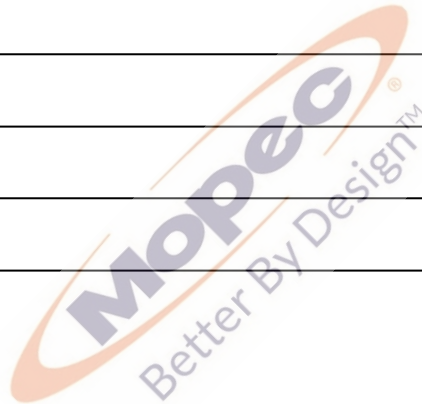

**SERVICE AND SUPPORT:**



21750 Coolidge Highway • Oak Park, MI 48237 USA  
+1 800-362-8491 • 248-291-2040 • Email: info@mopec.com

## **TABLE OF CONTENTS**

<b>IMPORTANT INFORMATION</b>	<b>1</b>
<b>TABLE OF CONTENTS</b>	<b>2</b>
<b>INTRODUCTION</b>	<b>3</b>
<b>FEATURES</b>	<b>4</b>
<b>UNPACKING</b>	<b>5</b>
<b>INSTALLATION</b>	<b>6</b>
<b>CLEANING AND MAINTENANCE</b>	<b>7</b>
<b>WARRANTY</b>	<b>8</b>
<b>ISO CERTIFICATE</b>	<b>9</b>
<b>UNIT DRAWING</b>	<b>10</b>



## **INTRODUCTION**

Designed to fit in a 30" front to back countertop (cabinet optional).

Removable work surface provides for easy cleaning. Downdraft work cell has a 3-1/2" cup drain for any overflow and wash downs.

Using downdraft fume exhaust technology the BF600 downdraft work cell provides a safe economical downdraft working environment. Ideal for working with reagents, formalin, xylene and alcohol. Requires a sink base cabinet.



## **FEATURES**

- Dimensions: 27" L x 27" W x 8" H
- Designed to fit in a 30" Front to Back Countertop
- Removable Perforated Work Surface
- BF600 has a 3-1/2" cup drain for any overflow and washdown

**Cabinet/Countertop not Included**



**Please note: BF600 is not an actual sink and should not be plugged to retain fluids.**

## **UNPACKING**

- 1.) Carefully inspect the exterior of the shipping container before opening. If the crate is damaged and the product has sustained damage immediately contact Mopec and the freight carrier. **Never discard the shipping container even if it is damaged beyond recognition.**
- 2.) Have the delivery driver note any suspected damage on the bill of lading and sign it. Mopec will help assist in filing a claim for product repair and/or replacement.
- 3.) Carefully open the containers and inspect the goods for concealed damage. If concealed damage is noticed (i.e. Broken welds, dented stainless, scratches, etc.) Follow through as noted above, and again do not discard the shipping material, they are important in settling claims.



## **INSTALLATION**

### **PROCEDURE**

The Sink is designed to be placed in a countertop opening and has mounting studs and braces under the table. The edges should be sealed to ensure water does not get trapped between the sink and counter top during clean ups.



**Shown as a Modified BF600 with a back splash and edge**

## **CLEANING AND MAINTENANCE**

All stainless steel surfaces can be cleaned with soap and water, which will remove debris.


The stainless steel surfaces can be disinfected with a non-caustic disinfectant.

**We highly recommend you DO NOT USE a bleach solution to clean your unit. Bleach will eventually erode stainless steel if not thoroughly rinsed.**

**The use of chlorine bleach will VOID THE STAINLESS STEEL WARRANTY.**

During the cleaning we suggest wiping the surface in the same direction as the satin finish, this will help elevate dirt from the grain finish. We suggest using BE045 PathCloud or BE047 Bench Wipe for cleaning purposes.

Scratches can be removed simply by using a "non-metallic" abrasive pad. Since most abrasive pads vary from one supply to another, we suggest rubbing the entire surface to blend the scratch and blend the balance of the surface.

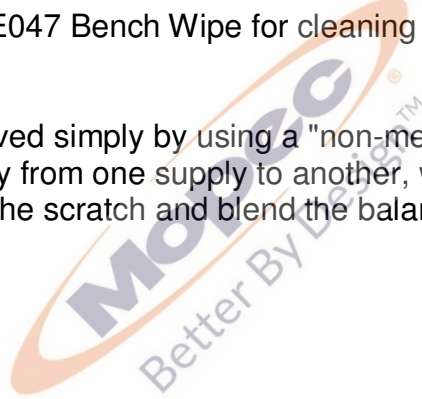



## **WARRANTY**

### **MOPEC BF600**

### **LIMITED WARRANTY**

Products manufactured by Mopec will be free from defects in material and workmanship and conform to Mopec's description or specifications. If a warranty claim is made within one (1) year from the earlier if the date of acceptance/first beneficial use, the defective or nonconforming Product or Part thereof will be repaired or (at Mopec's option) replaced free of charge, FCA Mopec's dock. All warranty claims must be in writing and received by Mopec within the warranty period. The warranty is not transferable (other than to customers of Mopec's authorized Distributors), and will not apply unless the Equipment has been properly installed, maintained and operated in accordance with all instructions; and does not apply to defects, nonconformities or other failure due to Equipment misuse, abuse, modifications, or other causes outside Mopec's control. If a warranty claim is made in writing within the warranty period, the defective or nonconforming Equipment (or Part thereof) will be repaired or (at Mopec's option) replaced free of charge, FCA Mopec's dock.

**THERE ARE NO WARRANTIES WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE HEREOF. THE WARRANTY AS SET FORTH HEREIN IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

To the extent that Mopec is acting as a supplier of Products manufactured by a third party, the Products will be warranted only to the extent that they are warranted by their manufacturers and Buyer (or its customer) agrees to look solely to the Product manufacturer for all warranty claims.

*MOPEC, 21750 COOLIDGE HIGHWAY, OAK PARK, MI 48237*



# CERTIFICATE OF REGISTRATION

This is to certify that

**MP Acquisition, LLC DBA Mopec**

21750 Coolidge Highway, Oak Park, Michigan 48237 USA

operates a

**Quality Management System**

which complies with the requirements of

**ISO 9001:2008**

for the following scope of registration

**The registration covers the quality management system for the design, engineering, manufacturing and installation of equipment and distribution of supplies for morgue, pathology, histology and necropsy applications.**

Certificate No.: CERT-0078089

File No.: 1068177

Issue Date: March 11, 2014

Original Certification Date: April 9, 2008

Current Certification Date: April 7, 2014

Certificate Expiry Date: April 6, 2017

Chris Jouppi  
President,  
QMI-SAI Canada Limited

Samer Chaouk  
Head of Policy, Risk and Certification



ISO 9001



Registered by:  
SAI Global Certification Services Pty Ltd, 296 Sussex Street, Sydney NSW 2000 Australia with QMI-SAI Canada Limited, 20 Carlson Court, Suite 200,  
Toronto, Ontario M9W 7K6 Canada (SAI GLOBAL). This registration is subject to the SAI Global Terms and Conditions for Certification. While all due care  
and skill was exercised in carrying out this assessment, SAI Global accepts responsibility only for proven negligence. This certificate remains the property  
of SAI Global and must be returned to them upon request.  
To verify that this certificate is current, please refer to the SAI Global On-Line Certification Register: [www.qmi-saiglobal.com/qmi\\_companies/](http://www.qmi-saiglobal.com/qmi_companies/)



**SAI GLOBAL**

INFORM. INSPIRE. IMPROVE.

