

DH SERIES

AUTOPSY CARTS/TABLE



REFERENCE MANUAL

Rev 6-15-14 ©



21750 Coolidge Highway • Oak Park, MI 48237 USA
+1 800-362-8491 • 248-291-2040 • Email: info@mopec.com

IMPORTANT INFORMATION


MODEL NO.:

SERIAL NO.:

DATE OF PURCHASE:

DATE OF INSTALLATION:

INSTALLATION NOTES:

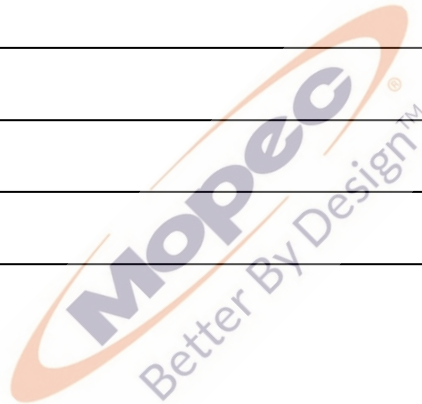

SERVICE AND SUPPORT:



21750 Coolidge Highway • Oak Park, MI 48237 USA
+1 800-362-8491 • 248-291-2040 • Email: info@mopec.com

TABLE OF CONTENTS

INTRODUCTION	4
CUSTOMER AND TECHNICAL SUPPORT	5
UNPACKING	6
CLEANING AND MAINTENANCE	7
STANDARD FEATURES	8
OPTIONAL FEATURES	9
WARRANTY	10



INTRODUCTION

The “DH” Series autopsy cart/table is used in conjunction with Mopec’s autopsy sinks. The frame is rigid with a built in slope allowing for fluid drainage into the autopsy sink. All Mopec carts now feature our exclusive, newly designed Total Lock caster system with non corrosive 8” heavy duty wheels. Caster wheel center is fabricated using state-of-the-art polyamide plastic with polyurethane treads for years of lasting service. Locking and braking are accomplished through stepping on the brake tabs. During transport, one caster swivel is locked for easy mobility.



CUSTOMER AND TECHNICAL SUPPORT

All Mopec staff members are Customer Service experts. We believe in providing nothing but the best possible service for all our customers. Our Technical Support group includes everyone from our design engineers to members of our production group. All are dedicated to providing comprehensive and courteous support to all Mopec customers.

The following information is included to help our customers communicate with Customer and Technical Support at Mopec.

Mopec

21750 COOLIDGE HWY.
OAK PARK , MICHIGAN 48237
TOLL FREE 800.362.8491
248.291.2040 PH. 248.291.2050 FX



UNPACKING

- 1.) Carefully inspect the exterior of the shipping container before opening. If the crate is damaged and the product has sustained damage then immediately contact Mopec and the freight carrier. **Never discard the shipping container even if it is damaged beyond recognition.**
- 2.) The receiver should note any suspected damage on the bill of lading and sign it. Mopec will help assist in filing a claim for product repair and/or replacement.
- 3.) Carefully open the containers and inspect the goods for concealed damage. If concealed damage is noticed (i.e. Broken welds, dented stainless, scratches, etc.) Follow through as noted above, and again do not discard the shipping material, they are important in settling claims.



CLEANING AND MAINTENANCE

DISINFECTING AND MAINTAINING STAINLESS STEEL SURFACES

All stainless steel surfaces can be cleaned with soap and water, which will remove debris.

The stainless steel surfaces can be disinfected with a non caustic disinfectant. We suggest using BE045 Path Cloud or BE047 Bench Wipe for cleaning purposes.

We recommend you **DO NOT USE** a bleach solution to clean your unit, bleach will eventually erode stainless steel if not thoroughly rinsed. **The use of chlorine bleach will VOID THE STAINLESS STEEL WARRANTY**

During the cleaning we suggest wiping the surface in the same direction as the satin finish which will help elevate dirt from the grain finish.

Scratches can be removed simply by utilizing a "non-metallic" abrasive pad and rubbing in the same direction as the satin finish.

Since most abrasive pads vary from one supply to another, we suggest rubbing the entire surface to blend the scratch and blend the balance of the surface.

STANDARD FEATURES

Removable Cart Top: Fabricated of 14 guage stainless steel with #4 satin finish. The top is recessed and dimpled toward the drain hole. A drain plug on a chain is provided. All exposed welds are ground smooth and grained to match a #4 satin finish

Cart Frame: Fabricated of 1-1/2" (3.8 cm) heavy gauge stainless steel tubing. All welding is continuous with an approximate 1/8" (.38 cm) radius bead and are clean and free of burrs and discoloration.

Welding: All welding is TIG type welding utilizing only stainless steel as a filler media. All weld fillets are approximately 1/4" (.64 cm) diameter. All welds are brushed clean removing any weld burns and burrs.

Wheels/Casters: Heavy-duty 8" diameter non corrosive polyamide wheels feature the Total Lock caster system.



OPTIONAL FEATURES

- **GA049 Autopsy Cart Top (Stainless Steel)** – Heavy- duty stainless construction.
- **GA849 Autopsy Cart Top (POLYETHYLENE BLUE)** – Mopec's X-Ray autopsy cart top is constructed out of solid sheet of Blue polyethylene x-rayed through with virtually no shadowing.



WARRANTY

MOPEC DH SERIES AUTOPSY CART

THE DA SERIES CART AND ALL PARTS THEREOF, MANUFACTURED BY MOPEC IS WARRANTED TO OUR CUSTOMERS TO BE FREE FROM DEFECTS IN MATERIAL AND CONSTRUCTION WHEN SUBJECTED TO NORMAL USE AND SERVICE. THIS WARRANTY SHALL NOT APPLY TO ANY EQUIPMENT OR PART THEREOF, WHICH HAS BEEN SUBJECT TO ALTERATIONS, ACCIDENT, MISUSE, OR WHICH HAS BEEN USED MORE THAN ITS PUBLISHED CAPACITY.

MOPEC'S OBLIGATION UNDER THIS WARRANTY IS LIMITED TO REPAIRING OR REPLACING, WITHOUT CHARGE ANY PART OR PARTS OF SAID EQUIPMENT WHICH PROVE DEFECTIVE AND WHICH OUR EXAMINATION SHALL DISCLOSE TO OUR SATISFACTION TO BE THUS DEFECTIVE. WE DO NOT ACCEPT RESPONSIBILITY FOR CONSEQUENTIAL DAMAGE RESULTING FROM SUCH DEFECTIVE PARTS. THE PERIOD OF WARRANTY ON MOPEC EQUIPMENT IS AS FOLLOWS:

ALL EQUIPMENT: EQUIPMENT IS WARRANTED IN ITS ENTIRETY FOR A PERIOD OF ONE (1) YEAR FROM THE DATE OF SALE. THIS WARRANTY INCLUDES PARTS AND LABOR FOR COVERED REPAIRS.

MOPEC WILL ACCEPT INCOMING DEFECTIVE PARTS, ONLY WHEN SHIPMENT IS PREPAID. THIS WARRANTY CONTRACT IS IN LIEU OF ALL OTHER WARRANTIES, AND RELEASES MOPEC OF ALL OTHER OBLIGATIONS OR LIABILITIES. IT NEITHER ASSUMES NOR AUTHORIZES ANY PERSON OR PERSONS TO ASSUME ANY OBLIGATION OTHER THAN THAT COVERED IN THIS WARRANTY.

MOPEC
21750 COOLIDGE HWY.
OAK PARK , MICHIGAN 48237
TOLL FREE 800.362.8491
248.291.2040 PH. 248.291.2050 FX



CERTIFICATE OF REGISTRATION

This is to certify that

MP Acquisition, LLC DBA Mopec

21750 Coolidge Highway, Oak Park, Michigan 48237 USA

operates a

Quality Management System

which complies with the requirements of

ISO 9001:2008

for the following scope of registration

The registration covers the quality management system for the design, engineering, manufacturing and installation of equipment and distribution of supplies for morgue, pathology, histology and necropsy applications.

Certificate No.: CERT-0078089

File No.: 1068177

Issue Date: March 11, 2014

Original Certification Date: April 9, 2008

Current Certification Date: April 7, 2014

Certificate Expiry Date: April 6, 2017

Chris Jouppi
President,
QMI-SAI Canada Limited

Samer Chaouk
Head of Policy, Risk and Certification



ISO 9001



Registered by:
SAI Global Certification Services Pty Ltd, 286 Sussex Street, Sydney NSW 2000 Australia with QMI-SAI Canada Limited, 20 Carlson Court, Suite 200,
Toronto, Ontario M9W 7K6 Canada (SAI GLOBAL). This registration is subject to the SAI Global Terms and Conditions for Certification. While all due care
and skill was exercised in carrying out this assessment, SAI Global accepts responsibility only for proven negligence. This certificate remains the property
of SAI Global and must be returned to them upon request.
To verify that this certificate is current, please refer to the SAI Global On-Line Certification Register: www.qmi-saiglobal.com/qmi_companies/

 **SAI GLOBAL**
INFORM. INSPIRE. IMPROVE.