


| | | | | | | | | | | | | | | |
|---|---|-------------------------------------|-----------------------------------|--------|-----------|-------|------------|--------|-------------|--------|--------|---|-----------------------------|-----------------------------|
| <p>THIS DOCUMENT CONTAINS PROPRIETARY INFORMATION OF MOPEC, AND MAY NOT BE COPIED IN WHOLE OR PART, AND NO USE SHALL BE MADE OF IT OTHER THAN FOR WHICH IT HAS BEEN SUBMITTED WITHOUT THE WRITTEN PERMISSION OF A MOPEC. MANAGER</p> <p>MOPEC 2018</p> | <p>UNLESS OTHERWISE SPECIFIED ALL DIMENSIONS IN INCHES</p> <p>TOLERANCES</p> <table border="0"> <tr> <td>0.0</td> <td>± 0.2 in</td> </tr> <tr> <td>0.00</td> <td>± 0.10 in</td> </tr> <tr> <td>0.000</td> <td>± 0.005 in</td> </tr> <tr> <td>0.0000</td> <td>± 0.0005 in</td> </tr> <tr> <td>ANGLES</td> <td>± 0.5°</td> </tr> </table> <p>BREAK ALL CORNERS 0.063 X 45° ALL FILLET RADII R0.063 /0.125in INSPECT OVAL DIMENSIONS</p> | 0.0 | ± 0.2 in | 0.00 | ± 0.10 in | 0.000 | ± 0.005 in | 0.0000 | ± 0.0005 in | ANGLES | ± 0.5° |  <p>Better By Design™</p> <p>21750 COOLIDGE HWY OAK PARK, MI 48237 PHONE: 800-362-8491</p> | <p>DRAWN BY: D.MADGWICK</p> | <p>PART NUMBER: MB1060L</p> |
| | | 0.0 | ± 0.2 in | | | | | | | | | | | |
| | | 0.00 | ± 0.10 in | | | | | | | | | | | |
| | | 0.000 | ± 0.005 in | | | | | | | | | | | |
| 0.0000 | ± 0.0005 in | | | | | | | | | | | | | |
| ANGLES | ± 0.5° | | | | | | | | | | | | | |
| <p>DATE: April 24, 2019</p> | <p>DESCRIPTION: MAESTRO 60" LH</p> | | | | | | | | | | | | | |
| <p>REV DATE: 09/10/19</p> | <table border="1"> <tr> <td>SIZE: B</td> <td>FILE NAME: MB1060L ROUGH-IN_REV E</td> <td>REV: E</td> </tr> </table> | SIZE: B | FILE NAME: MB1060L ROUGH-IN_REV E | REV: E | | | | | | | | | | |
| SIZE: B | FILE NAME: MB1060L ROUGH-IN_REV E | REV: E | | | | | | | | | | | | |
| <p>DRAWN TO 3RD ANGLE PROJECTION</p>  | <p>SCALE:</p> | <p>DO NOT SCALE IF IN DOUBT ASK</p> | | | | | | | | | | | | |

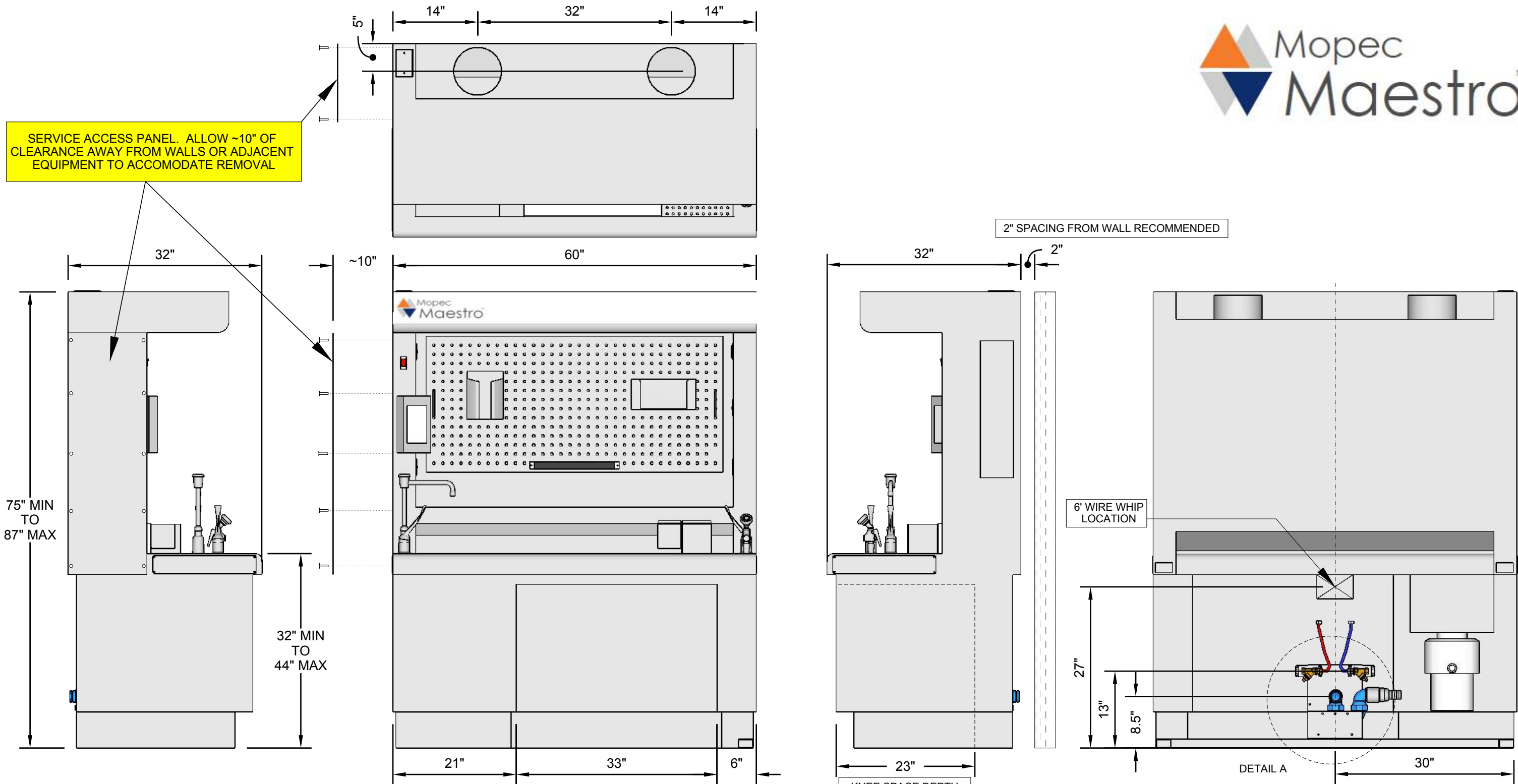

SERVICE ACCESS PANEL. ALLOW ~10" OF CLEARANCE AWAY FROM WALLS OR ADJACENT EQUIPMENT TO ACCOMODATE REMOVAL

2" SPACING FROM WALL RECOMMENDED

6' WIRE WHIP LOCATION

KNEE SPACE DEPTH

DETAIL A

30"

HVAC
CFM = 600
BUILDING VENTILATION DUCT STUBS PROVIDED BY OTHERS
*(2-EACH) 8" DIA. DUCT STUBS LOCATED AS SHOWN FROM IN-HOUSE VENTILATION (BY OTHERS)
*(2-EACH) 8" FLEXIBLE DUCTS PROVIDED BY MOPEC WITH HARDWARE FOR MOUNTING
*FLEXIBLE DUCTS ARE CUT AT 36" LONG FOR A TYPICAL CEILING CONNECTION HEIGHT OF 96". SPECIAL LENGTH FLEXIBLE DUCT CAN BE PROVIDED IF NECESSARY.
NOT REQUIRED IF RECIRCULATING OPTION IS ORDERED

ELECTRICAL
2 MULTIWIRED BRANCH 120V CIRCUITS
(1) 115/1/60, 20 AMPS, (REQUIRED)
(1) 115/1/60, 20 AMPS, (IF DISPOSAL IS DESIRED)
6' WIRE WHIP SUPPLIED BY MOPEC
DO NOT USE GFCI BREAKERS IN ELECTRICAL PANEL

| | |
|--|--------------------------------------|
| DESCRIPTION: MAESTRO 60" LH | FILE NAME: MB1060L ROUGH-IN_REV E |
| PART NUMBER: MB1060L | DESIGNER: D. MADGWICK |
| DATE: 09/11/19 | REV: E |
| <p>Better By Design™ 21750 COOLIDGE HWY OAK PARK, MI 48237 PHONE: 800-362-8491</p> | SCALE: DO NOT SCALE |

PLUMBING
1/2" NPT HOT (1/2" FLEX LINES)
STUB OUT FROM FACE OF WALL WITH 1/4" SHUT OFF VALVE PROVIDED BY OTHERS
1-1/2" DRAIN LINE
STUB OUT FROM FACE OF WALL 'P' TRAP PROVIDED AT DRAIN OUT BY MOPEC
1/2" NPT COLD (1/2" FLEX LINES)
STUB OUT FROM FACE OF WALL WITH 1/4" SHUT OFF VALVE PROVIDED BY OTHERS

DETAIL A