

Mopec *Maestro*

GROSSING STATION



EQUIP YOUR LABORATORY WITH
THE SAFEST & MOST PERSONALIZED GROSSING STATION

Mopec Maestro[®] Grossing Station

Mopec understands that each pathology professional has their own workspace preferences, and each laboratory holds their own safety, ergonomic and workflow requirements. With the Maestro's distinctive Ensemble Pegboard™, dozens of accessories and flexible safety options, every station can be quickly personalized to suit each individual.

PREMIUM WORKSURFACE

Cutting & rinsing surface

Each station features two removable panels with one perforated plate and one specimen cutting board with removable legs. The panels are capable of sliding across set rails for workspace flexibility and easy cleaning. The sink can also be adjusted for left or right handed users.

Height adjustability

Maestro workstations are ADA compliant, and they adjust to accommodate professionals of varying heights.



INTUITIVE CONTROLS & DEVICE MANAGEMENT

Easy-to-use touchscreen

With a quick tap, all Maestro features can be controlled using the new, integrated touchscreen. All user settings can be saved with individual user profiles to allow professionals to promptly adjust to their height and lighting preferences.

Wire containment

Four outlets are provided on each bench with three USB compatible and one GFCI. Cable covers are incorporated into the peg board and on the side of each station, therefore eliminating the look of messy cords.

TOP-NOTCH VENTILATION & SAFETY

Front air system

This patented, on-demand technology pushes air from the front of the cutting surface through the back of the area, creating a laminar flow pattern. This continuous ventilation protects the user from dangerous formalin exposure.

Safety notifications

On-screen warnings alert users of possible harmful conditions involving air flow, optional formalin collection and other hazards.

Optional Formalin Fume Detection

The Mopec Maestro[®] is the only grossing bench to offer patent pending formalin detection. This advanced, optional technology will measure the level of all VOC's, including fumes from the formalin being used on the grossing station and all ambient VOC's in the laboratory, reaching the sensor. It is intended as a tool to allow the user and the lab manager to understand the background VOC levels in the laboratory and to learn what activities cause these levels to elevate. It is not set to any specific safety standard, but rather is intended as an enhancement to an overall program of laboratory safety.



Harmonize Your Workspace with Optional Accessories

ENSEMBLE PEGBOARD™ ACCESSORIES

Movable options for precise workstation personalization. Add-on pegboard wings allow for more storage space.

- Organizer Bins
- Requisition File Holder
- Bar Code Holster
- Shelves
- Cell Phone Holder
- Magnetic Tool Holder
- Staining Dye Holder
- Specimen Scale Shelf
- Towel & Glove Box Holders
- SaniPath™ Wipe Dispenser
- Safety Glasses Holder
- Face Shield Holster
- Monitor Mount

MULTIPLE MONITOR MOUNTING OPTIONS FOR IMPROVED ERGONOMICS AND REDUCED NECK STRAIN.



OPTIONAL DRAWER SYSTEM

Storage options for quick reach accessibility


- Side Drawer
- Middle Drawer
- Small Over-the-Sink Drawer
- Instrument Soaking Inserts
- Keyboard Tray Insert
- Tissue Marking Dyes Insert



OTHER OPTIONS & ACCESSORIES

Additions for an enhanced workspace

- Cassette Labeling Shelf
- Side Monitor Arm
- Spatter-Free Formalin Dispenser
- Ventilated Trash
- CPU Bracket
- PathCam™ Imaging System
- Hand Spray
- Foot Pedal Faucet Control
- Infrared Faucet Control
- Safety Splash Shield
- LED Light Strip
- Magnifier
- Task Light
- Keyboard & Mouse Tray
- Keyboard Arm

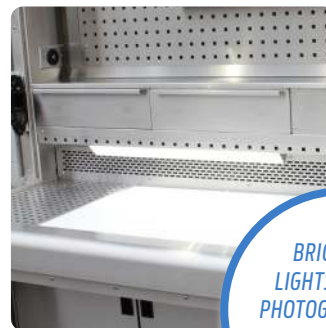

BRIGHT LED LIGHTSTRIP FOR PHOTOGRAPHY AND AN ILLUMINATED WORKSURFACE.

STANDARD FEATURES

Every Maestro comes with the following included:

- Dimensions: 60" L X 29" W
- Stainless Steel Construction
- Right-Hand Sink
- Backdraft Ventilation
- In-House Ventilation (Dual 8" Duct Stubs)
- Touchscreen Control Panel & Alert System
- On-Demand Front Air System (FAS)
- Seismic Anchoring Points
- Integrated Hood-Mounted Camera Bracket
- Faucet Mixing-Flow Control
- Ensemble Pegboard™ System
- Side Column Accessory Mounts
- Four GFCI-Protected Receptacles w/ USB
- One Perforated Grid Plate
- Sliding Work Surface White Cutting Board
- Two Wire Ports for Cord Management
- 1/2 hp Disposal
- Magnetic Instrument Bar
- One Large and One Small Organizer Bin
- Sanipath™ Wipes & Dispenser

Why choose the Mopec Maestro®?



Future Flexibility

Should laboratory space or staffing needs change, Maestro benches can be altered within your facility. Mopec technicians have the ability to change the sink location, add pegboard wings or easily replace parts, among other tasks.

Quality Construction

Each order is thoroughly tested and rigorously inspected against the highest industry standards. Every unit is issued with a standard one year warranty.

Knowledgeable Representatives

Mopec understands that every facility is unique. Our customer service and sales representative teams are educated in common facility challenges and can assist in customizing your equipment.

Complete Customization

Available in seven different sizes, with over 30 modification options, and a growing inventory of work station accessories, the Mopec Maestro is built to be customizable to your laboratory needs and individual preferences.

Mopec Maestro® Grossing Stations are ideal for:

Hospital Pathology
Private Pathology
Animal Research

Universities
Histology
Government Research

Made in the United States | Worldwide Shipping | Over 25 Years of Experience

DESIGN YOUR OWN MOPEC MAESTRO ONLINE AT
MOPEC.COM/MAESTRO

Mopec
BETTER BY DESIGN

www.mopec.com
800.362.8491
21750 Coolidge Hwy
Oak Park, MI 48237