



# MOPEC USER MANUAL

HOW TO ATTACH YOUR BD080 BONE DUST VACUUM COLLECTOR TO YOUR BD040 MOPEC 1000 AUTOPSY SAW





## WHAT YOU NEED

- BD080 Bone Dust Collector Vacuum
- Attachable Vacuum Nozzle and Adapter Bracket
- BD040 Mopec 1000 Autopsy Saw and Autopsy Blade
- Tools for Assembly

## INSTRUCTIONS

1. Make sure your Mopec 1000 Autopsy Saw and BD080 Bone Dust Vacuum are turned off and unplugged. Remove the autopsy blade from your saw and set aside.
2. Turn the knurled screw on the nozzle in a counter-clockwise direction to remove the adapter bracket.



3. Install the adapter bracket on the saw with the screws provided.




4. Reinstall the blade on the saw.



5. Push the small end of the molded vinyl hose end over the round portion of the nozzle until a secure fit is obtained.



- Slide the nozzle over the blade and align the slot on the nozzle with the adapter bracket and secure with the knurled screw.

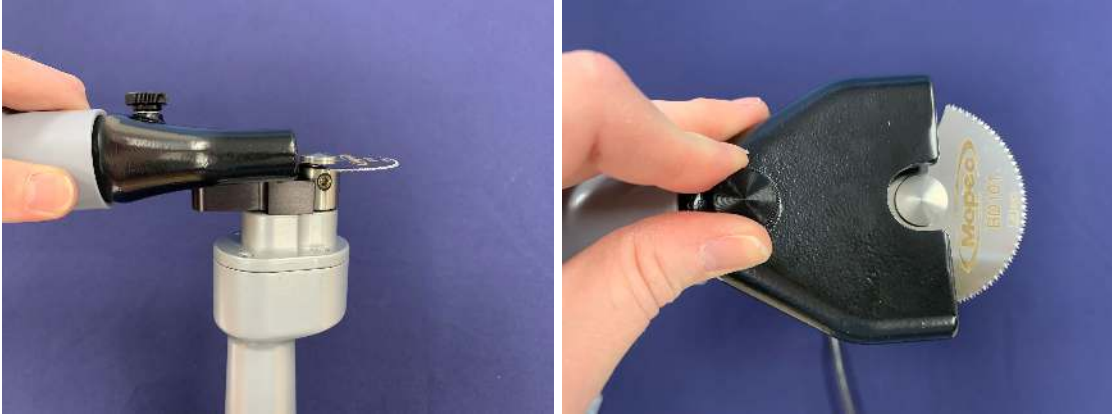

- The system is now ready for use.