

Instructions for Use

Harmony Tissue Marking Dyes, Amplify and Melody Spray

BG100, BG100-50, BG10*-15, BG10*60, BG10*-250, BG300, BG300-50

A coloured solution intended to be used to permanently mark the surgical margins and/or other general areas of interest on a tissue biopsy specimen; it is not intended to stain a tissue section for the visualization of intra/extracellular elements. It is one of a set, or a set of multiple colours (e.g., red, blue, black, green, violet and/or orange) which can withstand tissue processing (e.g., tissue fixation, freezing) without fading or bleeding. Applicators for precise marking or spreading of the dye onto the tissue specimen may be included.

Ready to use solution. Shake before use.

For laboratory use.

Keep in a cool, dry, well-ventilated area. Keep containers tightly closed. Store in correctly labelled containers.

Please see expiry on product label.

WARNING: Never apply a Harmony Tissue Marking Dye to a living person as doing so may cause permanent colouration.

INSTRUCTIONS FOR USE

1. For optimal results, fresh tissue should be patted dry prior to applying dyes. For fixed tissue blot excess fixative from the surface before dye application.
2. Apply Harmony Tissue Marking Dye to sample and remove excess dye.
3. Replace cap after use.
4. Allow Harmony Tissue Marking Dye to dry for 1 to 3 minutes at 15 to 30 degrees.

NOTE: For fatty tissue it is recommended to use Amplify Tissue Marking Dye Setting Solution. Ensure that the stained area is covered adequately. Leave dry for 1 minute. Replace cap after use.

NOTE: If using orange, red or yellow dyes it is recommended to use Melody Shielding Spray. Ensure that the stained area is covered adequately. Leave to dry for 1 minute. A white coating will appear when dry, this has no affect processing and will be removed during the processing steps. Replace cap after use.

5. Dispose of empty bottle in line with government guidelines.



Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.




Melody Spray Contains

Acetone: 25 - <50%

Ethanol: 25 - <50%

Methanol: 1 - <5%

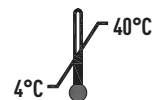

Mopec, 800 Tech Row,
Madison Heights, MI 48071
Toll Free: (800) 362-8491 | mopec.com

Mopec®

BETTER BY DESIGN™