

**Mopec** *MERC System*  
MORTUARY ENHANCED REMAINS COOLING  
*3/4 - 2 tons*



**INSTRUCTION MANUAL  
MERC SYSTEM**

**3/4 ton • 1 ton • 2 ton**

**www.mopec.com  
800.362.8491**

**800 Tech Row  
Madison Heights, MI 48071**



**INSTRUCTION MANUAL**  
**MERC SYSTEMS**  
**3/4 ton • 1 ton • 2 ton**

COVERING

**INSTALLATION**  
**OPERATION**  
**MAINTENANCE**

---

## TABLE OF CONTENTS

<b>1.0</b>	<b>GENERAL</b>	<b>7</b>
1.1	Introduction	8
1.2	Unit location	8
1.3	Receiving Instructions	10
1.4	Efficiency	10
1.5	Clean air act	10
1.6	Fluid treatment	11
1.7	Components	11
<b>2.0</b>	<b>INSTALLATION</b>	<b>13</b>
2.1	General	14
2.2	Unit Location	14
2.3	To and From process connections	15
2.4	Air-cooled condenser	16
2.5	Electrical connection	17
<b>3.0</b>	<b>OPERATIONS</b>	<b>19</b>
3.1	General	20
3.2	Start up/operations procedure	20
3.3	Instrument/operation	23
3.4	Shut down procedure	26
<b>4.0</b>	<b>TROUBLESHOOTING</b>	<b>27</b>
4.1	Unit Will Not Start	28
4.2	Compressor Hums But Will Not Start	28
4.3	Shuts Off On High Pressure	28
4.4	Shuts Off On Low Pressure	30
4.5	Compressor Shuts Off On Internal Overload	30
4.6	Low Or No Process Pressure Or Water Flow	30
4.7	Cooling Capacity Inadequate	30
4.8	Sensor	30
4.9	Coolant Pump	31
4.10	Chiller Controller	31
<b>5.0</b>	<b>MAINTENANCE</b>	<b>33</b>
5.1	Warranty service procedure	34
5.2	Periodic preventative maintenance	34
5.3	Special maintenance	35
5.4	Checking the refrigerant charge	35

---

<b>6.0</b>	<b>COMPONENTS</b>	<b>37</b>
6.1	Fluid system	38
6.2	Refrigeration system	38
6.3	Options	40
<b>7.0</b>	<b>RELATED DRAWINGS</b>	<b>41</b>
7.1	Typical Electrical : 2 ton units	42
7.2	Typical Electrical : 3/4 & 1-1/2 tons units	43
7.3	Mechanical Schematic	44
<b>8.0</b>	<b>APPENDIX</b>	<b>45</b>
8.1	Operations below 48°F	46
8.2	Refrigerant pressure temperature chart	48
8.3	Inhibited propylene glycol	49
8.4	Chiller capacity and derate chart	49

THIS PAGE INTENTIONALLY BLANK

---

## **1.0 GENERAL**

- 1.1** Introduction
- 1.2** Safety
- 1.3** Receiving Instructions
- 1.4** Efficiency
- 1.5** Clean Air Act
- 1.6** Fluid Treatment
- 1.7** Components

---

## 1.1 INTRODUCTION

- A. This manual covers portable chillers from 3/4 to 2 tons (2.53 to 7 kW) of cooling capacity using the M1 microprocessor control instrument and hermetic compressors. The standard fluid operating temperature range for this chiller is 20°F to 65°F for units using R404A or R-134A refrigerant. Units equipped with a CCPR valve operate up to 90°F. Units using other refrigerants have different standard operating ranges. Units operating below 48°F fluid require the use of a water/propylene glycol to prevent freezing. Customized units may have different operating ranges. Consult the factory if you have questions about the operating range of your chiller.
- B. The intent of this manual is to serve as a guide in the installation, operation and maintenance of your chiller. Improper installation can lead to equipment damage and poor performance. Failure to follow the installation, operation and maintenance instructions may result in damage to the unit that is not covered under the limited warranty. This manual is for standard products. The information contained in this manual is intended to be general in nature. The information is typical only and may not represent the actual unit purchased.
- C. Chemical refrigerants are used in this unit. The refrigerant is sealed and tested in a pressurized system however a system failure will release it. Refrigerant gas can cause toxic fumes if exposed to fire. Install this unit in a well-ventilated area away from open flames. Failure to follow these instructions may result in a hazardous condition. Recover refrigerant to relieve pressure before opening the system. See nameplate for refrigerant type. Do not use non-approved refrigerants or refrigerant substitutes.
- D. Customers should implement a refrigerant management program to document the type and quantity of refrigerant in each chiller. All refrigeration service technicians performing work on this chiller must be licensed and certified.
- E. When calling for assistance from the Manufacturer's Service Department, it is important to know the model and serial number of the particular unit. The model number includes critical unit information which is helpful when troubleshooting operating difficulties. The serial number allows the service team to locate manufacturing and testing records which can have additional information relating to a particular unit.

## 1.2 SAFETY

- A. It is important to become thoroughly familiar with this manual and the operating characteristics of the unit.
- B. It is the owner's responsibility to assure proper operator training, installation, operation, and maintenance of the unit.
- C. Observe all warning and safety placards applied to the chiller. Failure to observe all warnings can result in serious injury or death to the operator and severe mechanical damage to the unit.
- D. Observe all safety precautions during installation, startup and service of this equipment due to the presence of high voltage and refrigerant charge. Only qualified personnel should install, startup and service this equipment.



- E. When working on this equipment, observe precautions in literature and on tags, stickers and labels located on the equipment. Wear work gloves and safety glasses.



**WARNING:** This equipment contains hazardous voltages that can cause severe injury or death. Disconnect and lock out incoming power before installing or servicing the equipment.




**WARNING:** This equipment contains refrigerant under pressure. Accidental release of refrigerant under pressure can cause personal injury and or property damage. Exercise care while working on or around this equipment.



**WARNING:** Vent all refrigerant relief valves in accordance to ANSI/ASHRAE Standard 15, Safety Code for Mechanical Refrigeration. This equipment should be located within a well-ventilated area. Inhalation of refrigerant can be hazardous to your health and the accumulation of refrigerant within an enclosed space can displace oxygen and cause suffocation

#### Samples of Warning Labels applied to typical chillers.

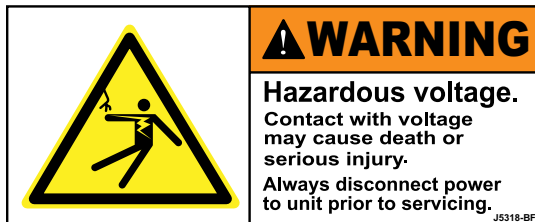

Alerts users to the danger of high voltage.



Alerts user to the danger of the rotating condenser fans on air condensed units.



Alerts user to the danger of the rotating condenser fans on air condensed units.



This symbol is seen on all chillers to alert user to the danger of the refrigeration system under pressure. System should only be serviced by a licensed technician.

---

### 1.3 RECEIVING INSTRUCTIONS

- A. Chillers are shipped skid mounted and wrapped in plastic prior to shipment. Check the overall condition of the equipment prior to accepting delivery.
- B. Check for visible damage and document any evident damage on the delivery receipt. Check the refrigerant gauges to be sure the system charge is intact. See the chart in section 8.4 for proper pressure readings based on the ambient temperature and refrigerant type used in the chiller. Shipping damage is the responsibility of the carrier.
- C. In order to expedite payment for damages, should they occur, follow proper procedures and keep detailed records. Take photographs of any suspected damage.

### 1.4 EFFICIENCY

- A. Long term efficiency of operation is largely determined by proper maintenance of the mechanical parts of the unit and the water quality. The Manufacturer recommends filtering the process water to prevent solids from plugging critical parts. The Manufacturer highly recommends that the services of a qualified water treatment specialist be obtained and their recommendations be followed. The Manufacturer accepts no responsibility for inefficient operation, or damage caused by foreign materials or failure to use adequate water treatment.



**WARNING:** Vent all refrigerant relief valves in accordance to ANSI/ASHRAE Standard 15, Safety Code for Mechanical Refrigeration. This equipment should be located within a well-ventilated area. Inhalation of refrigerant can be hazardous to your health and the accumulation of refrigerant within an enclosed space can displace oxygen and cause suffocation.

### 1.5 CLEAN AIR ACT

- A. Units manufactured after January 1, 2010 may contain refrigerant HFC-410A, HFC-407C, HFC-404A or HFC-134A. Most units manufactured prior to January 1, 2010 contain refrigerant HCFC-22.
- B. It is unlawful for any person in the course of maintaining, servicing, repairing, or disposing of refrigeration equipment to knowingly vent or otherwise dispose of any substance used as a refrigerant in the manner which permits such substance to enter the atmosphere.
- C. Very small releases associated with good faith attempts to recapture, reclaim or recycle such substance shall not be subject to the prohibition set forth in the preceding paragraph.
- D. Customers should implement a refrigerant management program to document the type and quantity of refrigerant in each chiller. All refrigeration service technicians performing work on this chiller must be licensed and certified.
- E. Vent all refrigerant relief valves in accordance to ANSI/ASHRAE Standard 15.




**WARNING:** Improper water treatment will void unit warranty.

## 1.6 FLUID TREATMENT

- A. The use of untreated or improperly treated water in a portable chiller may result in scaling, erosion, corrosion, algae or slime.
- B. It is recommended that the services of a qualified water treatment specialist be engaged to determine what water treatment is required.
- C. Advantage assumes no responsibility for equipment failures which result from untreated or improperly treated water.
- D. Do not use straight deionized or RO (reverse osmosis) water in this unit. Some customized units may be compatible with deionized or RO water. Consult the factory before using deionized or RO water.

## 1.7 COMPONENTS



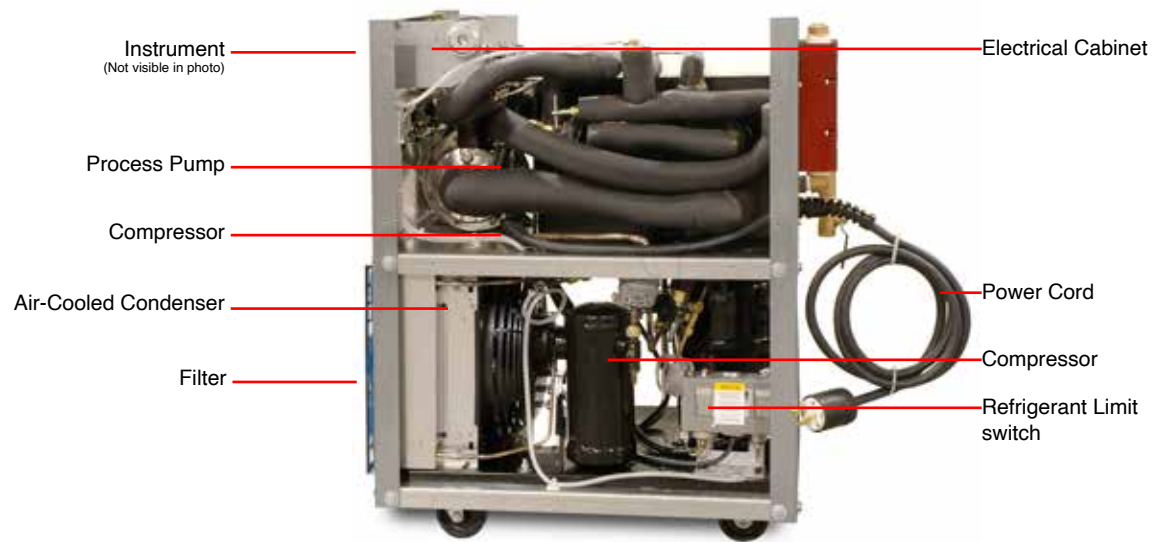

3/4 ton portable chiller

---

<b>2.0</b>	<b>INSTALLATION</b>
2.1	General
2.2	Unit Location
2.3	To and From Process Connections
2.4	Air-Cooled Condenser Installation
2.5	Electrical Connection

---

## 2.1 GENERAL

- A. Chillers are shipped skid mounted and wrapped in plastic prior to shipment. Check the overall condition of the equipment prior to accepting delivery. Check for visible damage and document any evident damage on the delivery receipt. Shipping damage is the responsibility of the carrier.
- B. All process piping materials (such as hose, rigid piping, valves or filters) used in process water piping circuitry must be rated for 100°F minimum temperature and 100 PSI minimum pressure.
- C. All such materials must have the equivalent or larger diameter of the particular process connection that length of process water piping is connected to.

## 2.2 UNIT LOCATION

- A. **Foundation.** The chiller must be installed on a rigid and level mounting surface with adequate strength to support the operating weight of the chiller including the weight of water and attached piping.

**This unit will contain water or water/glycol when operating. Locate the chiller where an unforeseen fluid leak will not cause damage to the surroundings or install the unit in such a way that an unforeseen fluid leak will not damage its surroundings.**

- B. For air-cooled and water-cooled models:
  - 1. These units are designed for indoor use only.
  - 2. For most efficient operation, locate the chiller in a level, clean, dry and well ventilated environment.
- C. **For air-cooled models:**
  - 1. The unit has an air-cooled refrigerant condenser. For air-cooled condensers, a motor driven fan generates air flow through the condenser to remove heat from the refrigerant system. The air cooled condenser on the unit will discharge approximately 15,000 BTU's per hour per ton of cooling and the air flow rate is approximately 1,000 cfm/ton.
  - 2. The unit must have a minimum entering air temperature of 60°F and a maximum entering air temperature of 90°F for efficient operation.
  - 3. The unit must have a minimum of **four** feet clearance at the air intake and **six** feet at the vertical exhaust air discharge and must have adequate ventilation to dissipate the rejected heat. The unit must not be placed in non-ventilated areas.
  - 4. Units with a motor-driven fan may not use duct work unless a booster fan of equal volume rating has been installed in the duct.

## 2.3 TO AND FROM PROCESS CONNECTIONS

- A.** Use appropriately rated hose or fluid piping to connect the process piping going to your process to the “to process” connection on the chiller.
- B.** Use appropriately rated hose or fluid piping to connect the process piping coming back from your process to the “from process” connection on the chiller.
- C.** Install a high volume basket strainer in the From Process line with isolation valves. A basket strainer or bag filter with a mesh screen of 20-40 microns will protect the unit. **A “wye” type strainer is not recommended because it does not have adequate debris holding capacity.**
- D.** Process water piping should be designed to avoid excessive elbows and/or lengths of pipe or hose. If using flexible hose avoid tight twists or curls and excessive lengths.
- E.** Valves and filters may be installed in the process water piping to facilitate service and maintenance provided that such devices maintain the full inside diameter of the process connection. If installed, all such devices must be open and clean during unit operation.
- F.** It is important to check all external process connections for leaks prior to starting the unit. It is also important to check external connections for leaks and internal hose clamps and connections for leaks after servicing the unit.

To Process


This label marks the To Process connection

From Process

This label marks the From Process connection



Typical wye strainer - Not recommended.

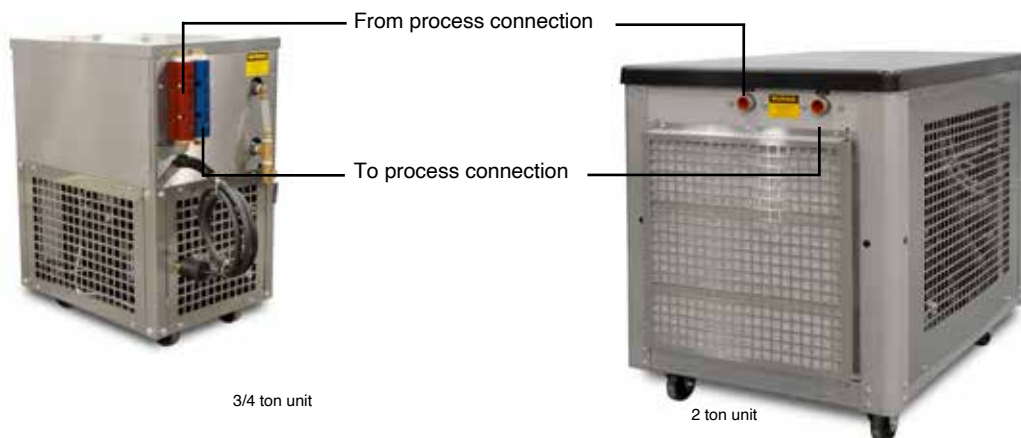

## 2.4 AIR COOLED CONDENSER

- A. Air-cooled condensers require ambient air temperatures between 60°F and 90°F for efficient operation. Operating above 95°F may result in elevated condensing pressures and eventual shut-down on the high pressure limit switch. Air temperatures below 60°F may result in over condensing pressures and cause a low pressure refrigerant fault. Consult with the Manufacturer's service department for more information on operating with ambients air temperatures above 90°F or below 60°F.
- B. Air flow is generated by the motor mounted fan. Air flow is from the outside of the chiller, through the condenser and exhausted through the unit. Exhaust air can not be ducted on motor mounted fan models.



Note: The 1-1/2 and 2 ton models include low ambient controls to -20F.

- C. A free air space of at least two (2) feet is required at the condenser intake and six (6) feet at the condenser discharge to allow for proper air flow.
- D. At full load, the chiller will discharge approximately 15,000 BTU's per hour per ton of cooling.
- F. **Condenser Air Filtering.** Use a filter that produces very low air flow restrictions. Generally a disposable fiberglass filter with a MERV rating of 2 - 3 provides adequate air filtration. The purpose of the filter is to prevent large particles such as dust, lint and debris from fouling the condenser. You should be able to see through the filter media. Using restrictive filter media can cause loss of chiller capacity, difficulty running in warm weather and fan motor overheating. Standard on certain models - optionally available on others.



Typical Filter materials.



Typical Filter shown on 3/4 ton air-cooled unit. Unit shown with optional pressure gauges.





**WARNING:** Do not connect the unit to a voltage supply not equal to the unit's voltage requirements as specified on the unit's data plate. Use of incorrect voltage will void the unit's warranty and cause a significant hazard that may result in serious personal injury and unit damage.



**WARNING:** Electric Shock Hazard. High Voltage is present in the electrical cabinet. Disconnect power before servicing. Follow all facility lock-out tag-out procedures.



**WARNING:** Check that all electrical connections are tight before starting. Disconnect power before servicing. Follow all facility lock-out tag-out procedures.

## 2.5 ELECTRICAL CONNECTION

### A. STANDARD MODELS

1. All electrical wiring must comply with local codes and the National Electric Code.
2. Electrical power supply requirements for standard units are identified on the equipment data tag. Determine that the plant's voltage supply is the same as the unit's voltage requirements, taking into account the SSCR Rating.
3. A factory installed power cable is supplied on most models. If a power cable is not supplied from the factory the customer must supply the proper power cable. The power cable (factory or customer supplied) will be connected to a customer supplied fused disconnecting means. The fused disconnecting means shall be sized and installed according to the unit's power supply requirements and local electrical codes. (Some custom units may include a fused or non-fused disconnect switch.)
4. A unit specific electrical drawing is shipped with the unit.
5. Voltage supplies must be within +/- 10% of the name plate voltage and must be within 2% from leg to leg. Extreme voltage imbalance or using the wrong voltage can damage your chiller and cause premature unit failure as well as a safety risk.
6. A proper ground is required for the unit.



Typical data tag.

### B. CONTROL CIRCUIT WIRING

1. For 3/4-ton & 2-ton units, the supplied control circuit is 110 volt, 1 phase, 60 cycle. For 1-ton models, the control circuit is 230 volt, 1 phase, 60 hz cycle.

- 
2. 220/240/1/50 hz models use 110v 1 phase 50 hz control circuit.
  3. Models with a factory installed transformer include a control circuit fuse and/or GFCI.

**C. GENERAL**

1. Make certain all ground connections to the unit are properly affixed.
2. Make certain power conductor, disconnecting means, and fusing are properly sized according to the unit's power supply requirements.
3. Follow all local and national codes.
4. Make certain that all owner and factory wire connections are tight before applying power to the unit.

Typical Electrical Panel : 3/4 ton units



Typical Electrical Panel : 2 ton units



- 
- 3.0 OPERATIONS**
- 3.1** General
  - 3.2** Start Up/Operations Procedure
  - 3.3** Instrument
  - 3.4** Shut Down

---

## 3.1 GENERAL

- A. Failure to follow the factory required operations procedure may adversely affect the unit's ability to adequately control process temperature and may create a hazardous operating condition which may result in serious operator injury and/or unit damage.



**WARNING:** Follow all Factory operations procedures. Failure to do so may create a hazardous operating condition which may result in serious operator injury and/or unit damage.

- B. The OPERATIONS segment of this manual is divided into the following sections:

- 3.2 **Start up/operations** - follow this segment to start the unit after the initial installation to the process system or to restart the unit after reinstallation to the same or different process system. This section includes information on system fill, electric motor phasing (motor rotation) and process flow adjustments.
- 3.3 **Chiller Control** - follow this segment to start up and operate the chiller control. This section includes information on setpoint selection and adjustment, and feature explanations.
- 3.4 **Shut down procedure** - follow this segment to shut down the unit. This segment includes information on system shut down, electrical power supply precautions, and disconnection from system.

## 3.2 START UP / OPERATION PROCEDURE

### A. System Fill

1. For operating temperatures from 48°F to 80°F use water to fill the unit. For operating temperatures below 48°F a water and propylene glycol mixtures must be used. An inhibited propylene glycol can be used for operating temperatures above 48°F if desired to prevent corrosion and scaling. Use the minimum ratio that is recommended by the glycol manufacturer. See section 8 of this manual for more information.
2. The unit has an internal reservoir which must be filled and maintained for proper operation.
3. **FLUID QUALITY CONTROL.** Lack of, as well as, improper fluid treatment can damage the chilling unit. The services of competent water treatment specialist should be obtained and their recommendations followed. It is the equipment owner's responsibility to prevent damage from foreign material or inadequate water treatment. See water treatment section in **section 1.6** of this manual for more information.
4. **MANUAL FILL:** Disconnect and lock out the electrical power supply and remove all necessary cover panels to access the reservoir. Add fluid directly to the reservoir. When the pump is first started, as process lines are filled and air is purged, additional fluid may be required to restore the reservoir to the correct level. Verify reservoir level via the coolant sight glass.

5. Do not use deionized water in this unit unless your unit was specifically designed for use with deionized water. Consult factory if not certain.

## B. PROCESS FLOW ADJUSTMENTS

1. The operator must determine and set proper water flow rate for the most efficient and trouble free operation.
  - a. Water flow rate through the process is determined by the pressure losses in the process loop. Generally, higher flow rates result in turbulent flow achieving maximum temperature control and lower maintenance. Since the evaporator in most liquid chillers is flow sensitive, the efficiency of operation is directly related to the flow of liquid.
  - b. Maximum chiller efficiency is obtained at approximately 2.4 gpm per ton of rated capacity. Low liquid flow can reduce efficiency and in some cases allow ice to develop in the evaporator which can damage the evaporator. Excessive liquid flow will trip the motor overload protection circuit.
2. Switch on the illuminated ON/OFF SWITCH to activate the process pump. Wait a few moments to allow air to be purge from system. Observe the COOLANT pressure gauge for steady readout. Check unit for low and high flow..
3. **LOW FLOW:** If a low flow condition is present, be sure all process valves are open. If all process valves are open and a low flow conditions exists, consider the following:

On the ¾-ton model, M1-.75A, it may be necessary to start and stop the chiller for priming the pump. Remove the stainless-steel top, remove the polyethylene tank cover, find the small copper air bleed line off the discharge of the pump, routed into the tank, toward the front of the chiller, and check for fluid flow. If on start the pump is not flowing fluid, stop then start. Repeat until flow is obtained.


  - a. Low flow can cause the low refrigerant pressure limit switch to trip.
  - b. To operate under a low flow condition, it is necessary to install a flow bypass system in the process circuitry. This will allow a portion of the flow to bypass the process and return directly to the chiller. This keeps the total flow above the cutoff point. The illustration below a typical bypass loop.



Typical reservoir.



Typical reservoir sight glass

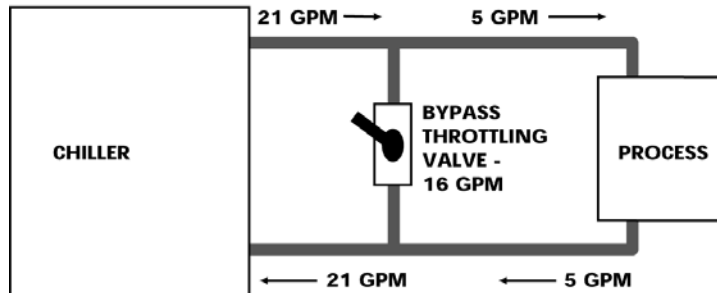

- c. Some models may have a factory installed bypass. Follow the instruction in paragraph “d” below to adjust the low flow bypass.

d. **Adjusting the Low Flow Bypass:**

**For Manual Low Flow Bypass Valves:** Start with the bypass completely closed and gradually open the valve until the low refrigerant pressure gauge reading is in the normal operating range for the refrigerant type used in the chiller.


**For Automatic Low Flow Bypass Valves:** A “T” handle or adjusting stem is located on the top of the valve. Turning the “T” handle or adjusting stem in the clockwise direction puts more pressure on the valve’s spring reducing bypass flow. Turning the “T” handle or adjusting stem counter clockwise puts less pressure on the spring and increases bypass flow. Adjust the “T” handle or adjusting stem until the low pressure gauge reading is in the normal operating range for the refrigerant type used in the chiller. If the low pressure gauge reading is below normal, reduce the pressure on the spring to provide more bypass.

4. **HIGH FLOW:** If a high flow condition is present:

- a. High flow can cause premature component wear and poor operating conditions.
- b. Adjust the flow so that an 8°F - 10°F rise in water temperature is indicated while the system is fully loaded.

---

### 3.3 INSTRUMENT/OPERATION

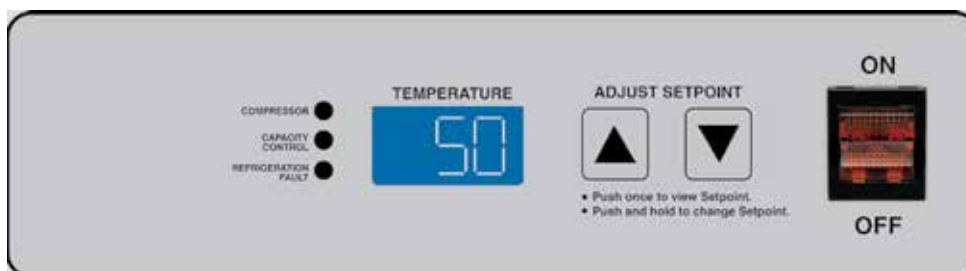

#### A. INSTRUMENT START-UP

1. When the correct electrical power and adequate water supply pressure are supplied to the unit, it is possible to start the unit.
2. Upon power up, the instrument displays “ChF” indicating that the unit is in Fahrenheit temperature mode or “ChC” indicating that it is in Celsius mode. The control then shows the current setpoint for approximately 2 seconds before reverting to the To Process temperature. When power is supplied to the unit, the ON/OFF switch will illuminate.

#### 3. PRECAUTIONS:

The chiller control is programmed from the factory with a setpoint range of 48° to 70°F. To operate below 48°F, the addition of inhibited propylene glycol and modification of the limit control settings are required. Diligent monitoring of the water/glycol solution is mandatory to prevent freezing of the evaporator. Freezing may cause the evaporator to rupture allowing water and freon to mix which will cause major damage to the refrigeration system.

R134A and R407C models operating above 70°F and R404A models operating above 60°F require the addition of a refrigerant crankcase pressure regulating (CPR) valve. The CPR valve is necessary to prevent overloading of the compressor which can cause premature failure. R410A models may be operated up to 80°F without a CPR valve.

Contact your local refrigeration contractor or the factory for further information. The operating range of the chiller control may be changed to 10°F - 90°F by adjusting the Setpoint Lockout (SPL) jumper. Refer to Section 3.3.c.3 of this manual for more information.

#### B. INSTRUMENT OPERATION

1. **To start the unit, toggle on the illuminated ON/OFF SWITCH.** The pump will start and the chiller control will begin temperature control operations.
2. To select setpoint temperature, press and hold the UP ARROW or DOWN ARROW keys until the desired set point temperature is displayed in the TEMPERATURE WINDOW. The default range for the setpoint temperature is 48° - 70°F or 9° - 21°C.

3. The setpoint temperature can be displayed by pressing the UP ARROW or DOWN ARROW keys. The setpoint temperature will be displayed for 5 seconds.
4. When the compressor is turned off, the instrument will wait 3 minutes before turning it back on regardless of the To Process temperature or setpoint. If a fault has occurred, the control will attempt to turn the compressor back on after 3 minutes. If the fault condition remains, the compressor will not restart and the control will retry after 1 minute. This sequence will repeat 3 times then the system will lock-out and the unit must be powered off/on to restart the unit.
5. Under normal conditions (no fault conditions, compressor has been off for three minutes) the instrument will turn on the compressor when the To Process temperature is above the setpoint. The instrument will engage the optional capacity control system (hot gas bypass) when the To Process temperature is below the setpoint by no more than 3 degrees.

The instrument will turn off the compressor and capacity control system when the To Process temperature is 4 degrees or more below the setpoint.

### C. CONTROL INSTRUMENT COMPONENTS

1. **ILLUMINATED ON/OFF SWITCH:** This rocker switch starts or stops the unit. Electrical power is supplied to the unit when the switch is illuminated.
2. **UP ARROW and DOWN ARROW KEYS:** Depress and hold this push button to increase (UP ARROW) or decrease (DOWN ARROW) the setpoint temperature. If the push button is pressed momentarily the setpoint value is incremented or decremented by one degree. If the push button is held down the setpoint will increase or decrease continuously.
3. **SETPOINT LOCK OUT JUMPER:** This jumper controls whether the user is allowed to reduce the setpoint below 48°F or 9°C. If the jumper is in position 1 (farthest from the SPL label) the user IS NOT ALLOWED to reduce the setpoint below 48°F or 9°C. If the jumper is in position 2 (closest to the SPL label) the user is allowed to reduce the setpoint to 10°F or -11°C.
4. **TEMPERATURE DISPLAY JUMPER:** If this jumper is in the “F” position, the To Process and Setpoint temperatures are displayed in Fahrenheit. If the jumper is in the “C” position, the To Process and Setpoint temperatures are displayed in Celsius.
5. **Probe Calibration:** This pot (CALPOT 1) is used to calibrate the probe circuit.






---

## D. STATUS LIGHTS

1. **COMPRESSOR:** Illuminates when compressor is turned on.
2. **CAPACITY CONTROL:** Illuminates when capacity control system is turned on.
3. **REFRIGERANT FAULT:** Illuminates when there is a high pressure or low pressure fault. Check troubleshooting section of this manual for more details.

**High Pressure Fault.** If the chiller control detects a high pressure condition it will immediately turn off the compressor and hot gas bypass.

**Low Pressure Fault.** After the compressor is turned on, the control has a 15 second buffer for the low pressure fault. If a low pressure condition occurs within the first 15 seconds, the control waits the amount of time specified by the "LP TIME" potentiometer before indicating an fault and turning off the compressor. If the condition is corrected before the time expires, no fault occurs. If a low pressure condition occurs 15 seconds after the compressor turns on, the instrument waits 20 seconds before indicating a fault and turning off the compressor.

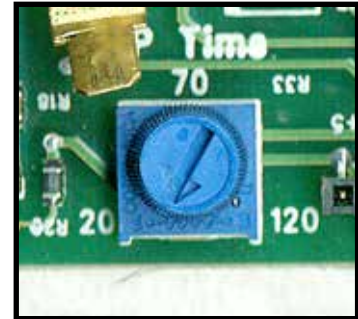

Low Pressure Potentiometer

## E. TEMPERATURE DISPLAY

1. A three digit display window indicates the appropriate temperature. The window also displays the numeric value for the setpoint temperature.
2. The To Process temperature is always displayed unless a button has been pressed. If there is a probe error, the display will show three dashes "---".
3. When the water temperature in the chiller is above approximately 100°F the display will show three dashes "---".

## F. PRESSURE GAUGES (optional)

1. **PROCESS PRESSURE GAUGE:** Indicates process pump pressure.
2. **REFRIGERANT HEAD PRESSURE GAUGE:** Indicates refrigerant pressure on the discharge side of the compressor. The refrigerant head pressure is also the condensing pressure which is critical to equipment efficiency.
3. **LOW PRESSURE GAUGE:** Indicates refrigerant pressure on the suction side of the compressor. This pressure will fluctuate with the process temperature.



Optional Pressure Gauges

---

### 3.4 SHUT DOWN/DISCONNECT SEQUENCE

#### A. PRECAUTIONS/WARNINGS

1. The operator must follow all shut down procedures outlined in this manual. If the operator fails to follow all procedures outlined in this manual, an unsafe condition can develop resulting in damage to the unit or personal injury.



**WARNING:** *Follow all shut down procedures outlined in this manual.*

#### B. UNIT SHUT DOWN

1. **To shut down the unit without disconnecting from the process:**
  - a. Move the ON / OFF switch to the off position.
2. **To shut down the unit and disconnect from the process:**
  - a. Move the ON / OFF switch to the off position.
  - b. Disconnect all process lines.

---

## **4.0 TROUBLESHOOTING**

- 4.1** Unit Will Not Start
- 4.2** Compressor Hums But Will Not Start
- 4.3** Shuts Off On High Pressure
- 4.4** Shuts Off On Low Pressure
- 4.5** Compressor Shuts Off On Internal Overload
- 4.6** Low or No Process Pressure or Water Flow
- 4.7** Cooling Capacity Inadequate
- 4.8** Sensor
- 4.9** Pumps
- 4.10** Chiller Controller



**WARNING:** Before troubleshooting or servicing this unit, follow all company lock-out tag-out procedures.

#### 4.1 UNIT WILL NOT START

- A. **Power off.** Check main disconnect.
- B. **Main line open.** Check fuses.
- C. **Loose terminals.** Tighten terminals with POWER OFF.
- D. **Control circuit open.** Check control voltage fuses and transformer.

#### 4.2 COMPRESSOR HUMS BUT WILL NOT START

- A. **Contactor problem.** Check contacts and contactor operation.
- B. **Low voltage.** Check voltage at main and at the unit. If voltage is OK at the main but low at the unit, increase wire size. If low at main, consult your local power company. Voltage must be +/- 10% nameplate rating.
- C. **No power on one phase of a three phase unit.** Check fuses in control panel and main disconnect. Also check unit wiring, main plant fuse and wiring. If the problem is with the main power supply coming into the plant, call the local power company.
- D. **Loose terminals.** Power off and follow all company lock-out tag-out procedure before tightening terminals.

#### 4.3 SHUTS OFF ON HIGH PRESSURE LIMIT

**Note:** Units are equipped with an adjustable high pressure switch. If the refrigerant pressure exceeds the setting of the adjustable switch it must be manually reset when the discharge pressure falls to a safe level.

- A. **Insufficient condenser air flow.** Check condenser filter for dirt, fins may be plugged with dirt or foreign material. Also, check for proper fan rotation.
- B. **Fan motor not operating.** Have electrician check fuses and wiring, motor starter and overloads, and motor. Repair or replace motor if defective.



Adjustable High Pressure Switch

---

## 4.4 SHUTS OFF ON LOW PRESSURE CONTROL

**Note:** Units are equipped with an adjustable low pressure switch. The low pressure switch will automatically reset when the pressure rises above the cut-in pressure. If this does not occur contact the Manufacturer's service department for instructions.



Adjustable Low Pressure Switch

If the unit low pressure limit is activated three (3) consecutive times during start-up the unit will lock out and will not be allowed to start. When the chiller control locks out due to low pressure after three (3) attempts, there is a problem that must be resolved before the unit should be restarted. To re-start, the unit must be powered down and back on. This should only be attempted after the cause of the low pressure condition has been found and the issue has been resolved. Repeatedly re-starting the unit will cause damage to the unit often including freezing and rupture of the chiller evaporator.

\* See Temperature-Pressure chart in Section 8.2 for refrigerant pressure.

The low pressure switch is set to cut-out at 10°F and cut-in at 15°F - 18°F\*. If a low pressure condition exists for more than five seconds the compressor will stop and a "L-P" fault will appear in the display window. Requires 30% PG Dowfrost or equal with a fluid freeze point rating ~10°F verified with a refractometer.

After the refrigerant pressure rises above the cut-in pressure, a three minute time delay will occur before the compressor restarts. This will protect the evaporator and compressor from damage should a problem occur in the refrigeration system or if the chiller is operated under circumstances which could cause damage to the refrigeration system.

- A. Low or no water flow through the evaporator.** Adjust the flow per section 3.2.C
- B. Head pressure too low.** M1-.75A models, check that entering air temperature is above 60°F. If below 60°F, find out reason why. Relocate M1-.75A to a room with ambient between 65F and 90F. M1-1A and M1-2A models are rated down to -20F ambient.
- C. Low refrigerant charge.** Check for adequate refrigerant charge (bubbles or misty sight glass indicates low charge). If charge is low, have system checked for leaks and recharged by a refrigeration serviceman.
- E. Improperly set low pressure switch.** Have a refrigeration serviceman reset control or replace if defective.
- F. Restriction in the liquid line.**
  - 1. Clogged filter drier.** Check for pressure or temperature drop and have drier core replaced by a refrigeration serviceman.
  - 2. Liquid line valve or suction valve on compressor is partially closed.** Open fully.
  - 3. Liquid line solenoid not opening fully or leaking during off cycle.** Have the solenoid repaired or replaced if defective by a refrigeration serviceman.
  - 4. Expansion valve plugged or inoperative.** Check thermal bulb and capillary tube for damage. Have repaired or replaced if defective by a refrigeration service man.

---

## 4.5 COMPRESSOR SHUTS OFF ON INTERNAL OVERLOAD

- A. **Control does not reset.** Have compressor windings and internal solid state safety control checked by a refrigeration serviceman. Have it repaired or replace if defective.

## 4.6 LOW OR NO PROCESS PRESSURE OR WATER FLOW

- A. **Valves.** Check if water valves are open.
- B. **Pump.** Check pump for correct rotation. Check pump suction for restriction. Replace motor if defective.
- C. **Filters.** Check filter in the chilled water circuit and clean if necessary.
- D. On the ¾-ton model, M1-.75A, it may be necessary to start and stop the chiller for priming the pump. Remove the stainless-steel top, remove the polyethylene tank cover, find the small copper air bleed line off the discharge of the pump, routed into the tank, toward the front of the chiller, and check for fluid flow. If on start the pump is not flowing fluid, stop then start. Repeat until flow is obtained.

Air Vent Purge Line (¾ ton models only). Check for flow through 1/8" copper tube. If no flow, clear or replace copper tube. Check for flow.

## 4.7 COOLING CAPACITY INADEQUATE

- A. **Low refrigerant charge.** Check for adequate refrigerant charge (bubbles or misty sight glass indicates low charge). If charge is low, have system checked for leaks and recharged by a refrigeration serviceman.
- B. **Hot-gas bypass valve stuck open.** Have repaired or replace if defective by a refrigeration serviceman.
- C. **Expansion valve plugged or inoperative.** Check thermal bulb and capillary tube for damage. Have repaired or replaced if defective by a refrigeration serviceman.
- D. **Plugged filter.** Check filter in chilled water circuit and clean.
- E. **Air in system.** Purge air.

## 4.8 SENSOR

- A. The sensor is a solid state temperature transducer which converts temperature input to proportional current output.
- B. To quickly test for a defective probe, switch connections between the defective probe and a probe known to be working properly. A defective sensor will display a "---" in the display window on the instrument control. Please note that "---" will also display when process temperatures are above 100°F.



Typical chilled water sensor probe with 2 pole connector.

---

## 4.9 COOLANT PUMP

- A. The vane pump is designed to operate at a specific flow and pressure at the maximum run load amp draw of the motor. Too little flow can overload the motor and cause the overload circuit to open and stop the pump.
- B. If the overload trips, check for electrical shorts, loose wires, or blown fuses. If these check OK, reset the overload circuit and restart the chiller.
- C. Check the amp draw and if overloaded, open the bypass valve until the amp draw drops to the proper level.

## 4.10 CHILLER CONTROLLER

- A. The control instrument is used for all normal set ups, diagnostics, temperature readout and operational information. It contains the software and electronic components which operate the control instrument.
- B. The control instrument is not field repairable. It can be easily removed and replaced or repaired if a problem occurs.
- C. All control instruments used in this water chillers are covered by the machine's warranty. Proprietary "tailor made" instrument are manufactured specifically for this unit.

If you experience problems with the control instrument, it's as easy as 1-2-3 to execute our repair or replacement system in order to get your equipment running.

THIS PAGE INTENTIONALLY BLANK



---

## **5.0**

### **MAINTENANCE**

- 5.1**    Warranty Service Procedure
- 5.2**    Periodic Preventative Maintenance
- 5.3**    Special Maintenance
- 5.4**    Checking the Refrigerant Charge

---

## 5.1 WARRANTY SERVICE PROCEDURE

- A. In the event of a problem with a chiller that can not be resolved by normal troubleshooting procedures, the customer is invited to consult the Service Department for assistance. The correct model number and serial number of the chiller must be available. The service department will attempt to isolate the problem and advise repair procedures. Often times, with the customer's input and with the machine diagnostics, problems can be determined with "over-the-phone" consultation.
- B. If the problem is beyond the scope of "over-the-phone" consultation, and if the warranty status of the machine is valid, the Manufacturer will contact the nearest authorized service contractor and provide authorization to conduct an "on-site" inspection of the unit in order to determine the course of repair. If the chiller is not covered by the warranty, the Manufacturer will advise on the repair and recommend available service contractors.
- C. It is of the utmost importance that you provide the correct model number and serial number of the machine in question. This will allow the Service Department to obtain the correct manufacturing records which will help to properly troubleshoot the problem and obtain the proper replacement parts when they are required. This information is stamped on the data tag that is attached to the electrical enclosure of each machine.
- D. The Service Department must be notified prior to any repair or service of a warranty nature. Warranty claims will not be honored without prior authorization.

## 5.2 PERIODIC PREVENTATIVE MAINTENANCE

- A. Lubricate all motors. Note that some motors are supplied with sealed bearings.
- B. Tighten all wire terminals.
- C. Clean and check motor starter and contactor contacts.
- D. Check safety switch settings.
- E. Clean condenser fins of dust and dirt (air cooled models only).
- F. Back flush evaporator.
- G. Check glycol/water solution ratio for operating temperature.
- H. Check system for leaks.
- I. Refrigerant sight glass: Check for bubbles when compressor is operating at 100%. Check the moisture indicator for a color other than green.
- J. Clean unit.

---

### 5.3 SPECIAL MAINTENANCE

- A.** Any service of the refrigeration system must be accomplished by a certified refrigeration technician.
1. Addition of compressor oil.
  2. Addition of refrigerant.
  3. Repair of a refrigerant leak.
  4. Adjustment of super heat.
  5. Changing of filter-drier or drier core.
  6. Repair of a refrigeration solenoid.

### 5.4 CHECKING THE REFRIGERANT CHARGE

- A.** All standard chillers are manufactured with thermostatic expansion valves as the metering device to the evaporator.
- B.** All standard chillers have a refrigerant sight glass (figure 5.4A) with a moisture indicator. To check the refrigerant charge under normal operating conditions:
1. Remove the plastic cap covering the sight glass.
  2. Start the chiller and allow system pressures and temperatures to stabilize.
  3. With the unit operating at 100% capacity (not in the “capacity control” mode) the sight glass should appear clear with no foam or bubbles evident. If foam or bubbles are evident, the chiller has suffered from a loss of refrigerant and should be checked by a qualified refrigeration technician.
  4. The “dot” in the middle of the sight glass is the moisture indicator. It should appear green at all times. A white or yellow color indicates moisture has invaded the refrigeration system, which is detrimental to the life of the compressor. The filter-drier should be replaced by a qualified refrigeration technician.



Refrigerant Sight Glass

THIS PAGE INTENTIONALLY BLANK

- 
- 6.0 COMPONENTS**
    - 6.1** Fluid System
    - 6.2** Refrigeration System
    - 6.3** Options

---

## 6.1 FLUID SYSTEM

- A. **MOTOR/PUMP ASSEMBLY:** The motor/pump assembly circulates chilled fluid to the process loop. The pump assembly is built of total stainless steel to maintain water quality.
- B. **RESERVOIR.** The vented reservoir is sized for the chiller application to support the flow rate. The reservoir provides a stable water temperature under varying load conditions.
- C. **LOW FLOW BYPASS:** If your process will experience lower than design flow rates or intermittent flow, a low flow bypass valve is factory included to maintain flow.



Typical Coolant Pump



Typical Reservoir

## 6.2 REFRIGERATION SYSTEM

- A. **COMPRESSOR:** Compressors take low pressure/low temperature refrigerant gas and compress the gas into high pressure/high temperature gas.
- B. **AIR COOLED CONDENSER:** The air cooled condenser removes heat from the compressed refrigerant gas. The action causes the gas to “condense” into a liquid state still under high pressure. Air flow across the condenser is achieved via a motor driven fan assembly or centrifugal blower.
- C. **FILTER-DRIER:** The filter-drier removes contaminants and moisture from the liquid refrigerant.
- D. **REFRIGERANT SIGHT GLASS:** The refrigerant sight glass indicates refrigerant charge and moisture content.
  - Full refrigerant charge is determined by a clear liquid flow.
  - Bubbles indicate low refrigerant.
  - Moisture content is indicated by the color of the element.
  - Element color is normally green.
  - If the color of the element is chartreuse or yellow, the system has been contaminated with moisture. In such case, the filter-drier must be replaced. The replacement of the filter-drier must be completed by a qualified refrigerant service technician.
- G. **EXPANSION VALVE:** The expansion valve throttles flow of refrigerant liquid into the evaporator and creates a pressure drop in the refrigerant system that allows the liquid refrigerant



Typical automatic low flow bypass valve.



Compressor



Typical Air-Cooled Condenser

---

to “boil off” inside the evaporator.

- H. EVAPORATOR:** The evaporator is a tube-in-tube or brazed plate design where the refrigerant liquid is allowed to evaporate (boil off) to absorb heat (BTU) from the process fluid. As the heat is absorbed, the process fluid is chilled.
- I. CAPACITY CONTROL SOLENOID:** The chiller is equipped with a capacity control feature. The capacity control feature matches the chiller capacity to the cooling load to provide temperature control and to prevent short cycling of the compressor. The hot gas bypass solenoid valve reduces the chiller capacity to about 50%.
- J. HIGH/LOW PRESSURE SWITCH:** The high and low pressure switches protect the refrigeration system from usage operating conditions. Units may be equipped with either an adjustable or non-adjustable ‘fixed’ high and low pressure switches.

The **high pressure switch** is factory set and protects the refrigeration components and personnel from potential damage and injury from excessive pressure. The high pressure safety must not be altered in the field for any reason (see section 8.1 for factory settings).

The **low pressure switch** is factory set to open at 32°F and to close at 36°F - 38°F\*. The low pressure switch protects the chiller from damage caused by low refrigerant pressure operating below the cooling fluid’s freeze point.

**NEVER LOWER THE CUT OUT SETTING WITHOUT ADDING GLYCOL TO THE CIRCULATING SYSTEM. EVAPORATOR DAMAGE WILL RESULT AND WILL NOT BE COVERED BY THE WARRANTY.**



Typical Filter Drier



Typical Expansion Valve



Typical Brazed Plate Evaporator



Adjustable high pressure switch



Adjustable low pressure switch

## 6.3 UNIT OPTIONS

- A. OVERHEAD PIPING KIT:** To avoid reservoir overflow during shut down periods from overhead piping, Advantage recommends the installation of an overhead piping kit. The overhead piping kit consists of a full line size solenoid valve in the “from process” line that is interlocked with the pump and a check valve (single direction valve) in the “to process” line.
- C. PROCESS LINE SHUT OFF VALVES:** These valves are full size ball valves installed on the “to” and “from” process lines at the exit of the chiller. These valves can be used to adjust the process flow rate and to shut off the flow rate to isolate the chiller.
- D. E-STOP BUTTON:** The Emergency stop button is a mushroom type button which will shut down all chiller operations when engaged.
- E. NON-FUSED OR FUSED DISCONNECT SWITCH:** On board disconnect switch provides a convenient location for isolating all electric power to the chiller.
- F. AUDIBLE AND OR VISUAL ALARM:** Alarms indicate out-of-spec operating conditions. There are two types of alarms:
- Audible Alarm: A simple buzzer alarm emits a loud alarm. Standard on certain models.
- Visual Alarm: Emits an audible sound and flashes a beacon.
- G. CONDENSER SCREEN:** Optional for air-cooled models only. The condenser screen is a filter for the air-cooled condenser to prevent air borne solids and debris from clogging the condenser. Use a non restrictive filter material with a MERV rating of 2 - 3. Standard on certain models.
- H. OPTIONAL LOWER PANELS.** Lower panels cover the exposed areas of the unit to hide and protect the refrigerant and coolant components. Standard on certain models.



Typical overhead piping kit. Shown with optional process line shut off valves.



Emergency stop button. Shown with Power On light.



Non-fused disconnect show.



Typical Alarm Beacon.




---


## **7.0 RELATED DRAWINGS**

- 7.1** Typical Electrical : 2 ton units
- 7.2** Typical Electrical : 3/4 & 1-1/2 ton units
- 7.2** Typical Mechanical


### 7.1 Typical Electrical : 2 ton units

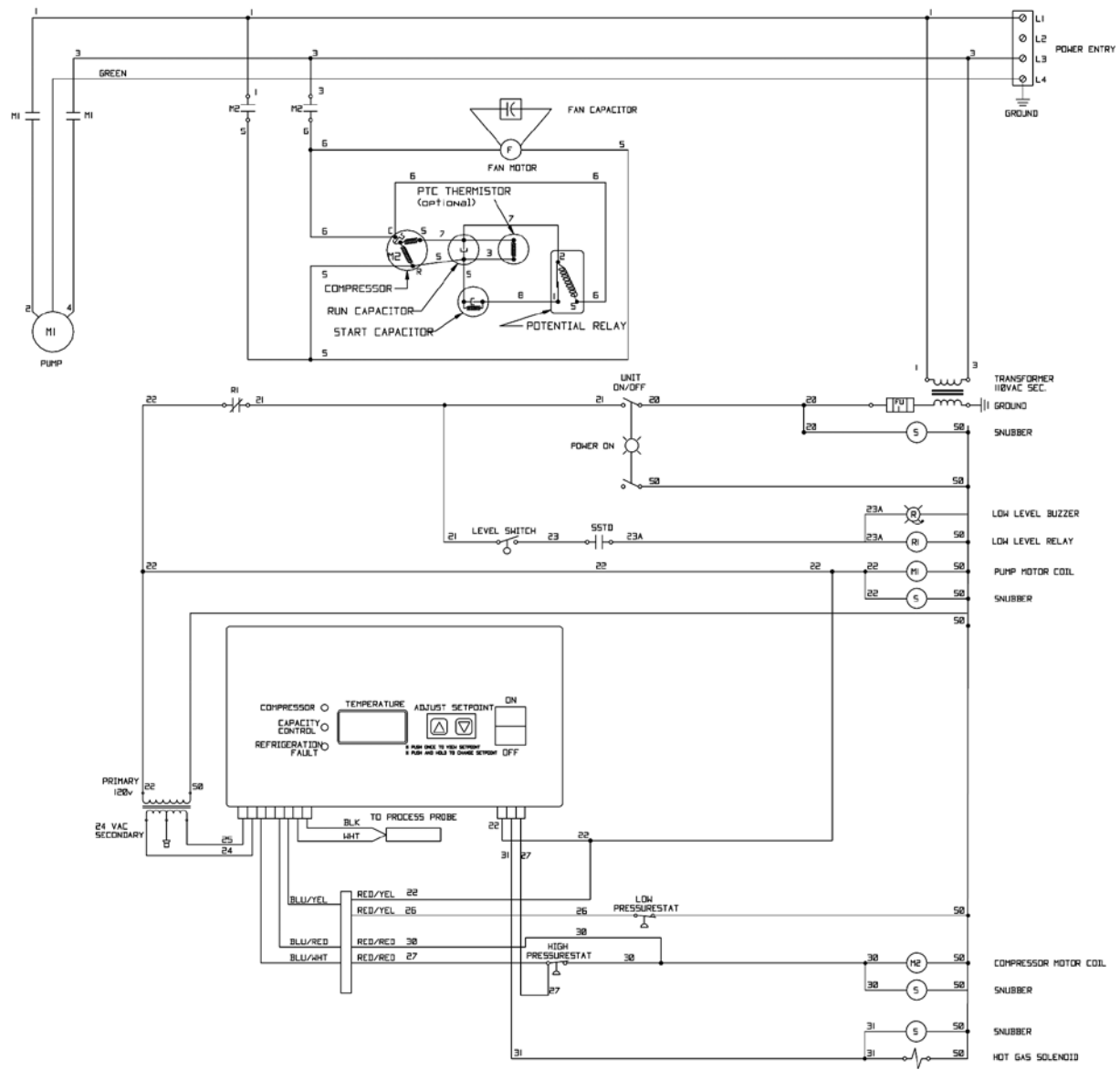

## 7.2 Typical Electrical : 3/4 & 1-1/2 ton units

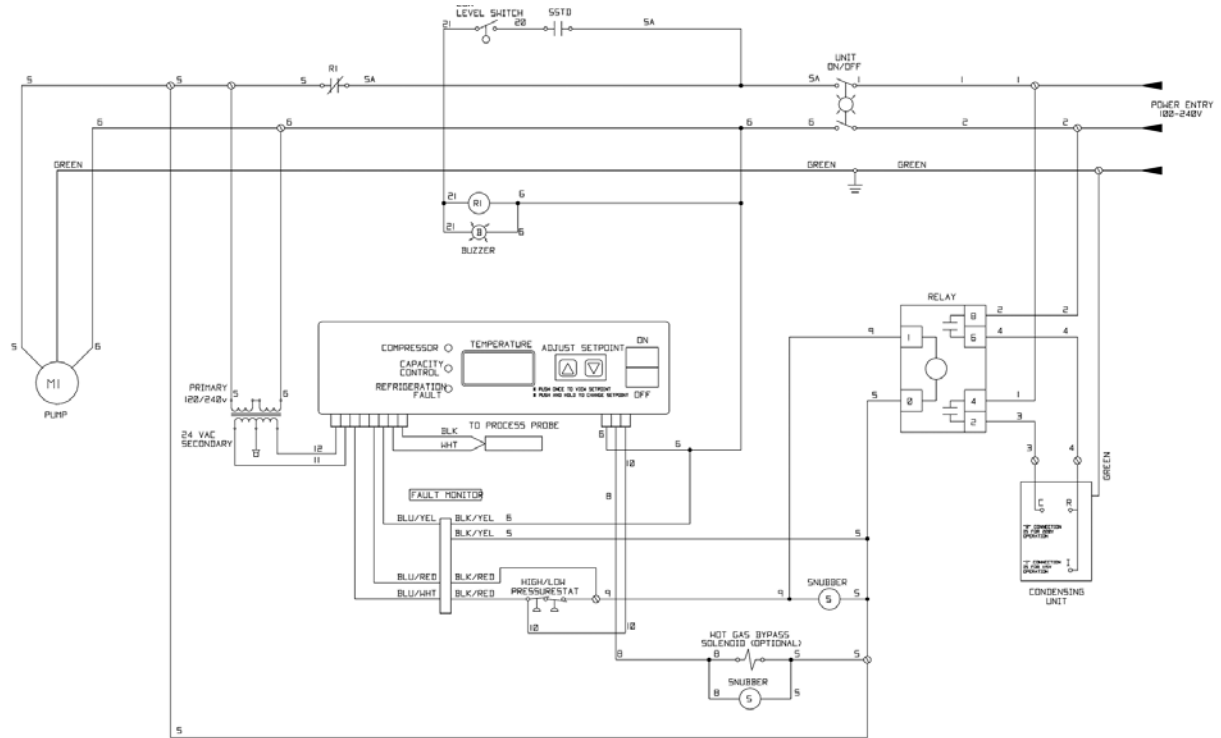

### 7.3 Typical Mechanical

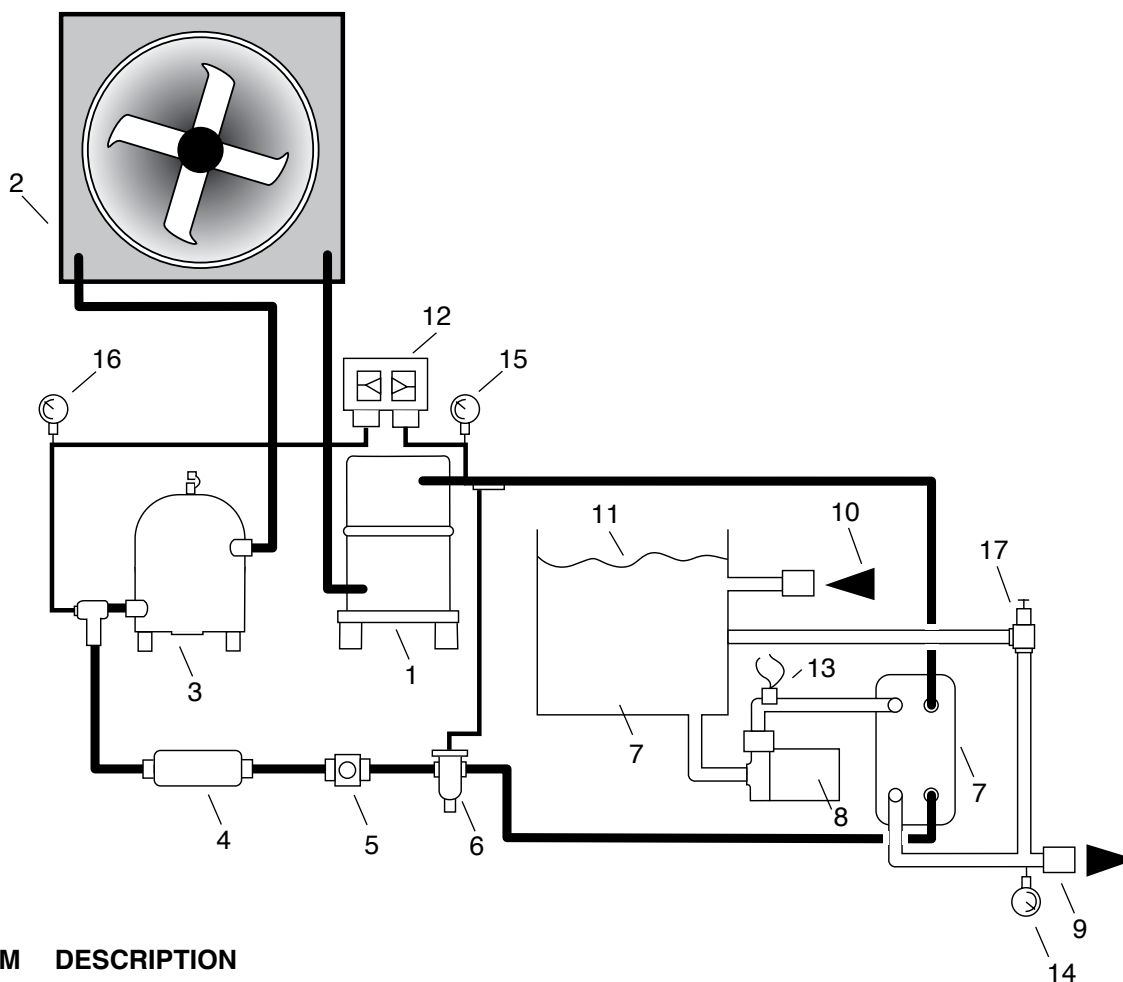

ITEM	DESCRIPTION
1	Compressor
2	Air cooled condenser
3	Liquid receiver
4	Filter-drier
5	Refrigerant sight glass
6	Expansion valve
7	Evaporator
8	Pump
9	To process connection
10	From process connection
11	Reservoir tank
12	Refrigerant safety switch
13	Control Sensor
14	To process pressure gauge
15	Low refrigerant pressure gauge
16	High refrigerant pressure gauge
17	Low flow pressure bypass valve

---

## **8.0 Appendix**

- 8.1** Operations Below 48°F
- 8.2** Refrigerant Pressure-Temperature Chart
- 8.3** Inhibited Propylene Glycol
- 8.4** Chiller Capacity and Derate Chart Mechanical Schematic

## 8.1 OPERATIONS BELOW 48°F FLUID OR 38°F AMBIENT

- A. The chiller is never to be operated below 48°F leaving water temperature without several precautionary measures. All controls are factory adjusted for 48°F and above operations.
- B. Before readjusting the protective devices, a satisfactory antifreeze solution must be substituted for the recirculating chilled water. This mixture will consist of inhibited propylene glycol and water. Do not substitute an inhibited propylene glycol and water solution with common automotive type antifreeze. The chart on the next page outlines the glycol percentages at various water temperatures.
- C. Fluid must be tested with a refractometer to verify proper glycol percentages for freeze protection. The ratio shall be according to the chart below. Too much glycol can cause capacity and control problems.
- D. **DO NOT USE AUTOMOTIVE TYPE ANTI-FREEZE.**
- E. **Once a satisfactory antifreeze solution is in place the protective devices may be adjusted and the control instrument can be unlocked to allow operation below 48°F.**
- E. If your unit is equipped with an adjustable low pressure switch,

**Never lower the cut out setting on the adjustable low pressure switch without adding glycol to the circulating system. Evaporator damage will result and this damage is not covered by the factory warranty.**



Adjustable low pressure switch

- F. **ADJUSTING THE SETPOINT LOCK OUT JUMPER:** The control instrument has a setpoint lock out jumper. This jumper controls whether the user is allowed to reduce the setpoint below 48°F or 9°C. If the jumper is in position 1 (farthest from the SPL label) the user IS NOT ALLOWED to reduce the setpoint below 48°F or 9°C. If the jumper is in position 2 (closest to the SPL label) the user is allowed to reduce the setpoint to 10°F or -11°C. To operate below 48°F leaving fluid temperature the jumper will need to be in position 2.

Setpoint Lock Out Jumper



- G. Once all safety provisions are made, the temperature control set point may now be lowered to the desired operating temperature.
- H. **WARNING:** do not use any type or brand of automotive antifreeze. Automotive antifreeze contains corrosion inhibitors - silicates - designed for compatibility with the materials in automotive engines. Unfortunately, silicates can gel and cause deposits to foul and insulate heat exchanger surfaces. In your chilling system that can mean higher energy costs, high pumping costs, and possibly even shut downs for system cleaning. We recommend the use of DowFrost or Monsanto DFS-1.

**Refrigerant Low Pressure Switch Cut-Out & Cut-In Settings**

Operating or Ambient Temperature	Glycol	Freeze Point	Cut Out Temp	Cut In Temp	R22		R134A		R410A	
					Cut-Out	Cut-In	Cut-Out	Cut-In	Cut-Out	Cut-In
48° - 70°F	0%	32°F	32°F	36°F - 39°F	58#	63#	28#	33#	102#	111#
25° - 47°F	30%	10°F	10°F	15°F - 18°F	33#	38#	12#	17#	63#	72#
10° - 24°F	40%	-5°F	-5°F	0°F - 7°F	20#	25#	4#	9#	43#	52#

Operating or Ambient Temperature	Glycol	Freeze Point	Cut Out Temp	Cut In Temp	R404A		R407C	
					Cut-Out	Cut-In	Cut-Out	Cut-In
48° - 70°F	0%	32°F	32°F	36°F - 39°F	72#	79#	52#	58#
25° - 47°F	30%	10°F	10°F	15°F - 18°F	44#	49#	28#	34#
10° - 24°F	40%	-5°F	-5°F	0°F - 7°F	29#	34#	16#	22#

**High Pressure Cut Out (maximum)**

Refrigerant	Air-Cooled	Water-Cooled
R22	380#	360#
R134A	260#	260#
R407C	405#	360#
R410A	610#	550#
R404A	405#	360#

## 8.2 REFRIGERANT PRESSURE-TEMPERATURE CHART

Temperature		Refrigerant				
°F	°C	R-22	R-410a	R-407c	R-134a	R-404a
-60	-51.1	11.9	0.9	16.0	21.6	-
-55	-48.3	9.2	1.8	13.7	20.2	-
-50	-45.6	6.1	4.3	11.1	18.6	-
-45	-42.8	2.7	7.0	8.1	16.7	-
-40	-40.0	0.6	10.1	4.8	14.7	4.9
-35	-37.2	2.6	13.5	1.1	12.3	7.5
-30	-34.4	4.9	17.2	1.5	9.7	10.3
-25	-31.7	7.5	21.4	3.7	6.8	13.5
-20	-28.9	10.2	25.9	6.2	3.6	16.8
-18	-27.8	11.4	27.8	7.2	2.2	18.3
-16	-26.7	12.6	29.7	8.4	0.7	19.8
-14	-25.6	13.9	31.8	9.5	0.4	21.3
-12	-24.4	15.2	33.9	10.7	1.2	22.9
-10	-23.3	16.5	36.1	11.9	2.0	24.6
-8	-22.2	17.9	38.4	13.2	2.8	26.3
-6	-21.1	19.4	40.7	14.6	3.7	28.0
-4	-20.0	20.9	43.1	15.9	4.6	29.8
-2	-18.9	22.4	45.6	17.4	5.5	31.7
0	-17.8	24.0	48.2	18.9	6.5	33.7
1	-17.2	24.8	49.5	19.6	7.0	34.7
2	-16.7	25.7	50.9	20.4	7.5	35.7
3	-16.1	26.5	52.2	21.2	8.0	36.7
4	-15.6	27.4	53.6	22.0	8.6	37.7
5	-15.0	28.3	55.0	22.8	9.1	38.8
6	-14.4	29.1	56.4	23.7	9.7	39.8
7	-13.9	30.0	57.9	24.5	10.2	40.9
8	-13.3	31.0	59.3	25.4	10.8	42.0
9	-12.8	31.9	60.8	26.2	11.4	43.1
10	-12.2	32.8	62.3	27.1	12.0	44.3
11	-11.7	33.8	63.9	28.0	12.6	45.4
12	-11.1	34.8	65.4	29.0	13.2	46.6
13	-10.6	35.8	67.0	29.9	13.8	47.8
14	-10.0	36.8	68.6	30.9	14.4	49.0
15	-9.4	37.8	70.2	31.8	15.1	50.2
16	-8.9	38.8	71.9	32.8	15.7	51.5
17	-8.3	39.9	73.5	33.8	16.4	52.7
18	-7.8	40.9	75.2	34.8	17.1	54.0
19	-7.2	42.0	77.0	35.9	17.7	55.3
20	-6.7	43.1	78.7	36.9	18.4	56.6
21	-6.1	44.2	80.5	38.0	19.2	57.9
22	-5.6	45.3	82.3	39.1	19.9	59.3
23	-5.0	46.5	84.1	40.2	20.6	60.6
24	-4.4	47.6	85.9	41.3	21.4	62.0
25	-3.9	48.8	87.8	42.4	22.1	63.4
26	-3.3	50.0	89.7	43.6	22.9	64.8

Temperature		Refrigerant				
°F	°C	R-22	R-410a	R-407c	R-134a	R-404a
27	-2.8	51.2	91.6	44.7	23.7	66.2
28	-2.2	52.4	93.5	45.9	24.5	67.7
29	-1.7	53.7	95.5	47.1	25.3	69.2
30	-1.1	54.9	97.5	48.4	26.1	70.7
31	-0.6	56.2	99.5	49.6	26.9	72.1
32	0.0	57.5	101.6	50.9	27.8	73.8
33	0.6	58.8	103.6	52.1	28.6	75.3
34	1.1	60.2	105.7	53.4	29.5	76.9
35	1.7	61.5	107.9	54.8	30.4	78.5
36	2.2	62.9	110.0	56.1	31.3	80.2
37	2.8	64.3	112.2	57.5	32.2	81.7
38	3.3	65.7	114.4	58.9	33.1	83.5
39	3.9	67.1	116.7	60.3	34.1	85.2
40	4.4	68.6	118.9	61.7	35.0	86.9
41	5.0	70.0	121.2	63.1	36.0	88.6
42	5.6	71.5	123.6	64.6	37.0	90.4
43	6.1	73.0	125.9	66.1	38.0	92.2
44	6.7	74.5	128.3	67.6	39.0	94.0
45	7.2	76.1	130.7	69.1	40.0	95.8
46	7.8	77.6	133.2	70.6	41.1	97.6
47	8.3	79.2	135.6	72.2	42.2	99.5
48	8.9	80.8	138.2	73.8	43.2	101.4
49	9.4	82.4	140.7	75.4	44.3	103.3
50	10.0	84.1	143.3	77.1	45.4	105.3
55	12.8	92.6	156.6	106.0	51.2	115.3
60	15.6	101.6	170.7	116.2	57.4	126.0
65	18.3	111.3	185.7	127.0	64.0	137.4
70	21.1	121.5	201.5	138.5	71.1	149.3
75	23.9	132.2	218.2	150.6	78.6	161.9
80	26.7	143.7	235.9	163.5	86.7	175.4
85	29.4	155.7	254.6	177.0	95.2	189.6
90	32.2	168.4	274.3	191.3	104.3	204.5
95	35.0	181.9	295.0	206.4	113.9	220.2
100	37.8	196.0	316.9	222.3	124.1	236.8
105	40.6	210.8	339.9	239.0	134.9	254.2
110	43.3	226.4	364.1	256.5	146.3	272.5
115	46.1	242.8	389.6	274.9	158.4	291.9
120	48.9	260.0	416.4	294.2	171.1	312.1
125	51.7	278.1	444.5	314.5	184.5	333.4
130	54.4	297.0	474.0	335.7	198.7	355.6
135	57.2	316.7	505.0	357.8	213.5	379.1
140	60.0	337.4	537.6	380.9	229.2	403.7
145	62.8	359.1	571.7	405.1	245.6	429.6
150	65.6	381.7	607.6	430.3	262.8	456.8
155	68.3	405.4	645.2	456.6	281.0	484.8



---

## 8.3 INHIBITED PROPYLENE GLYCOL

- A.** To operate liquid chillers below 48°F, it is necessary to add **inhibited propylene glycol** to the circulating system to lower the freeze point and prevent damage to the cooling system. Inhibited propylene glycol contains corrosion inhibitors which are compatible with most industrial heat transfer surfaces. Inhibited propylene glycol is manufactured by:
- Dow Chemical - "DowFrost" (1-800-258-2436)
  - Monsanto "Therminol FS" (1-800-459-2665)
  - Advantage Engineering "Thermofluid" (1-317-887-0729)
- B.** Automotive anti-freeze must never be used in industrial heat transfer applications. Automotive anti-freeze contains silicate type corrosion inhibitors designed to be compatible with automotive components. In an industrial application, the silicates will form a gel on the heat transfer surface which will result in substantial reduction in cooling capacity and is virtually impossible to remove.

## 8.4 CHILLER CAPACITY AND DERATE CHART

Standard chiller rating is at 50°F. For all other temperature settings, output tonnage is altered as follows:

**NOTES:**

If operation of the chiller at less than 48°F is required, an inhibited propylene glycol solution is required.

Consult factory for chiller operation below 20°F.

Ambient conditions affect air cooled chiller operation and capacity. Standard entering air temperature rating is 90°F for M1-.75A & M1-1A models and 95°F for M1-2A models. For ambient air conditions greater than noted, chiller derating will occur. For ambients over 90°F or 95°F consult factory.

OUTPUT TEMPERATURE °F	FULL AVAILABLE % CAPACITY
60	105%
50	100%
45	90%
40	80%
35	70%
30	60%
25	50%
20	40%

THIS PAGE INTENTIONALLY BLANK

END

© 2022 MANUFACTURER

RE 20220304