

10

11

REV	DESCRIPTION	DATE	EDN #	APPR
A	INITIAL RELEASE	00-XXX-19	XXXXX	MM

A

B

C

D

E

F

G

H

Item Number	Quantity	File Name	Title	Material
1	1	I46414-100-Aasm	MAST ASSEMBLY	
2	1	I46414-200-Aasm	CABINET	
3	1	I46414-900-Aasm	FOOT PEDAL ASSEMBLY	
4	1	I46414-1000-Aasm	COMPUTER SHELF	
5*	1	I46414-001-Apsm	BACKING PLATE	11 GA. TYPE T304 SS
6	1	I46414-002-Apar	USB ADAPTER MOUNT	UHMW POLYETHYLENE, 1" X 2" (NATURAL)
7	1	I46414-003-Apsm	STOP PLATE	11 GA. TYPE T304 SS
8	2	I46414-005-Apar	MOUNTING KNOB	ROD, STAINLESS STEEL, 1" DIA, TYPE 304
9	1	PM0765.asm	CAMERA ARM	
10	1	PM0354-Basm	KEYBOARD ARM	
11	1	PM0355-Basm	MONITOR ARM	
12	3	PD0059.par	CASTER TOTAL LOCK #028529	TENTE - 3477UARI25P30-13
13	1	PD0060.par	CASTER STEER LOCK #029560	TENTE - 3471UARI25P30-13 LEVER BK
14	1	PM0775.par	Plug Snap-in, for 2.76" to 2.84" Inside Sq.	
15*	2	PM0052.par	GROMMET - 1-1/2"ID. X 2-1/8"OD.	
16*	8	PM0009.par	BUMPER POLYURETHANE, ADHESIVE BACK	
17	4	PL0112.par	Paddle-Handle Push-to-Close Latch	
18*	5	PE0852.asm	USB OUTLET, 36"	
19*	1	PE0855.par	HDMI OUTLET	
20*	2	PE0856.asm	USB, DUAL, 120MM COOLING FAN	
21*	2	PE0853.asm	USB OUTLET, 72"	
22*	1	PL0115.par	SS Weld-on Pull Handle	
23*	1	PE0618.par	ETHERNET CONNECTOR	
24*	1	pe0115.par	CONNECTOR - IEC LINC CORD	
25	4	PB0705.par	SS Hex Head Screw, 1/4"-20 x 3-1/2"	
26*	2	PB0119.par	SCREW - SS, #8 x 3/4"L, TRUSS PHILLIPS	McMASTER-CARR # 93406A197
27	1	PE0857.asm	USB WIFI ADAPTER	
28*	1	PM0772.par	Webbing Guide	
29*	1	I46414-004-Apsm	OUTLET BOX	18 GA. TYPE T304 SS
30*	3	PB0039.par	SCREW - SS, 8-32 x 1/2"L, TRUSS PHILLIPS	McMASTER-CARR # 91770A194
31	14	PB0398.par	SCREW - SS, 4-40 X, 3/8" L ROUND HEAD PHILLIPS	McMaster-Carr#91773A108
32*	12	PB0125.par	SCREW - SS, 10-32 x 5/8"L, PAN PHILLIPS	McMaster-Carr#91772A830
33	2	PM0770.par	Grommet for 1" Hole Dia.	
34	1	PE0037.par	ROCKER SWITCH	
35*	4	PB0234.par	BOLT - ZINC, 1/2-13 x 1-1/2", HEX FULLY THREADED	McMASTER-CARR # 92865A716
36*	8	PB0706.par	#10 x 5/8", SS SHEET METAL SCREW	
37*	4	PB0027.par	SCREW - SS, 6-32 x 1/2"L, 82 DEG FLAT PHILLIPS	McMaster-Carr#91771A148
38*	2	PB0016.par	NUT - SS, 1/4"-20, NYLON LOCK	McMASTER-CARR # 91831A029
39*	8	PB0101.par	BOLT - SS, 3/8-16 x 1"L, HEX FULLY THREADED	McMASTER-CARR # 92240A624
40*	4	PB0103.par	WASHER - SS, 1/2", SPLIT LOCK	Stainless steel
41*	2	PB0338.par	#8-32 INSERT SPIN TITE	Stainless steel
42*	1	PE0510.par	RECEPTACLE GFCI, 20 AMP, 125V, Str Blade, Gray, Hospital	Stainless steel
43	1	S7972.asm	PIVOT ADAPTER / CABLE PASS THROUGH	
44	1	AX250HD-001.asm	HD CAMERA ENCLOSURE	
45*	1	AX250HR-001.asm	HR CAMERA ENCLOSURE	
46	2	PB0895.par	SS, SHCS, 5/16"-18 X 1-3/8" L	
47*	1	PE1006.par	OPEN/CLOSED BUSHING	
48*	1	PM0011.par	GROMMET	
49*	1	PE0782.par	Nylon Wall Plate, GFCI WALL PLATE	

1

2

3

4

5

6

7

8

9

10

11

12

13

14

15

16

17

18

19

20

21

22

23

24

25

26

27

28

29

30

31

32

33

34

35

36

37

38

39

40

41

42

43

44

45

46

47

48

49

MOPEC 2021

THIS DOCUMENT CONTAINS PROPRIETARY INFORMATION OF MOPEC, AND MAY NOT BE COPIED IN WHOLE OR PART, AND NO USE SHALL BE MADE OF IT OTHER THAN FOR WHICH IT HAS BEEN SUBMITTED WITHOUT THE WRITTEN PERMISSION OF A MOPEC MANAGER

UNLESS OTHERWISE SPECIFIED ALL DIMENSIONS IN INCHES

TOLERANCES

0.00 ± 0.1 in

0.000 ± 0.05 in

0.0000 ± 0.010 in

0.0000 ± 0.0010 in

ANGLES ±0.5°

BREAK ALL CORNERS 0.063 X 45°

ALL FILLET RADII R0.063 /0.125in

INSPECT OVAL DIMENSIONS

MATERIAL:

MOPEC GROUP

MOPEC MORTTECH

DRAWN TO 3RD ANGLE PROJECTION

DRAWN BY: dstewart2

DATE: 6/5/2025

CHECKED BY:

DATE: 6/5/2025

APPROVED BY: MMcCLAIN

DATE: 6/5/2025

PART NUMBER: AX260HD-HR

DESCRIPTION: AX260HD & HR

SIZE: A2

FILE NAME: AX260HD-HR.dft


SCALE: 1:5

DO NOT SCALE IF IN DOUBT ASK

SHEET 1 OF 4

REV: A



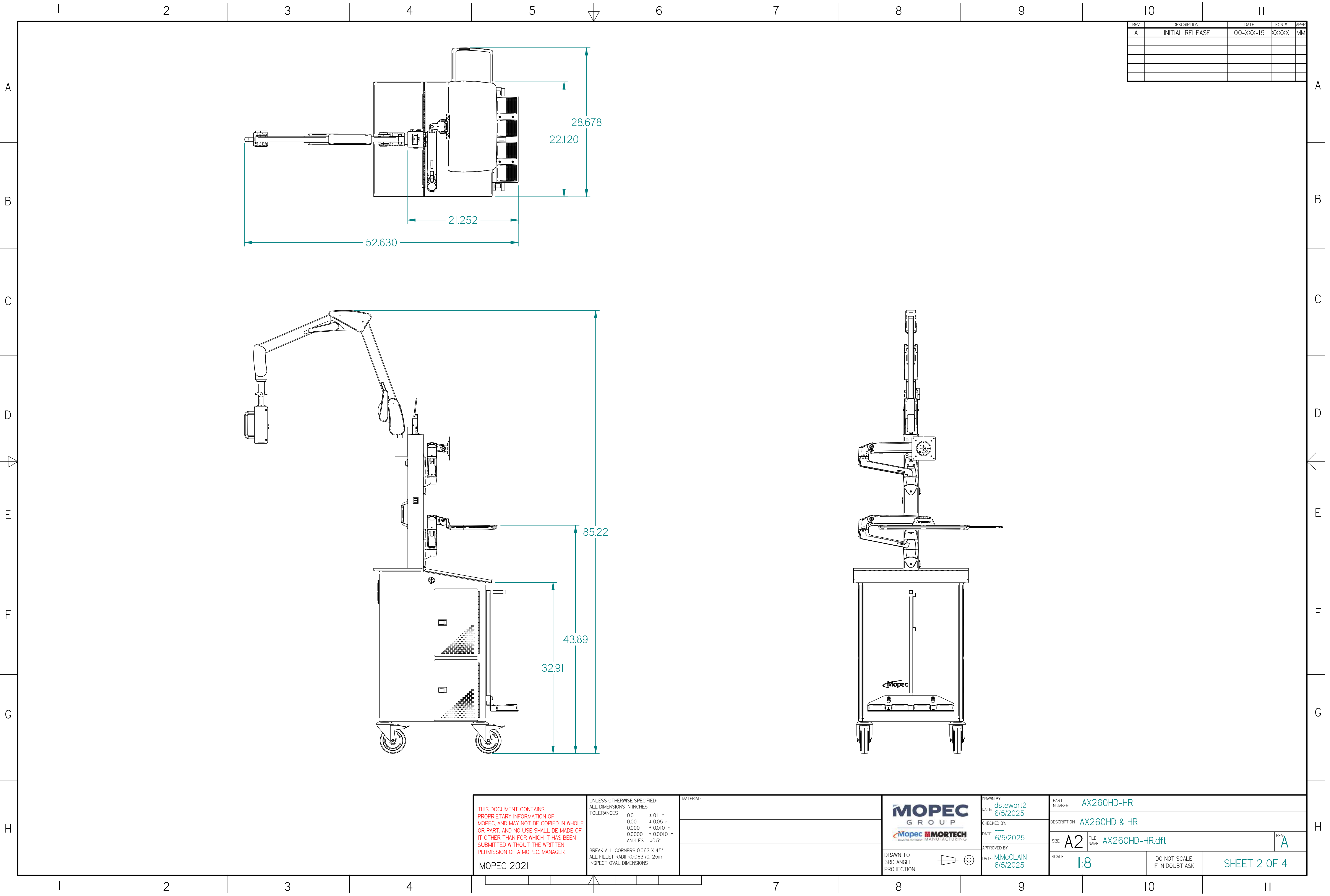

REV	DESCRIPTION	DATE	ECN #	APPR
A	INITIAL RELEASE	00-XXX-19	XXXXX	MM

THIS DOCUMENT CONTAINS
PROPRIETARY INFORMATION OF
MOPEC, AND MAY NOT BE COPIED IN WHOLE
OR PART, AND NO USE SHALL BE MADE OF
IT OTHER THAN FOR WHICH IT HAS BEEN
SUBMITTED WITHOUT THE WRITTEN
PERMISSION OF A MOPEC MANAGER

MOPEC 2021

UNLESS OTHERWISE SPECIFIED:
ALL DIMENSIONS IN INCHES
TOLERANCES 0.0 ± 0.1 in
0.00 ± 0.05 in
0.000 ± 0.010 in
0.0000 ± 0.0010 in
ANGLES ±0.5°
BREAK ALL CORNERS 0.063 X 45°
ALL FILLET RADI R0.063 /0.125in
INSPECT OVAL DIMENSIONS

MATERIAL


MOPEC
GROUP
Mopec MORTECH
MOBILE ROBOTICS TECHNOLOGY

DRAWN TO
3RD ANGLE
PROJECTION

DRAWN BY:
DATE: dstewart2
6/5/2025
CHECKED BY:
DATE: ---
6/5/2025
APPROVED BY:
DATE: MMcLAIN
6/5/2025

PART
NUMBER AX260HD-HR
DESCRIPTION AX260HD & HR
SIZE A2 FILE NAME AX260HD-HR.dft
SCALE 1:8 DO NOT SCALE
IF IN DOUBT ASK
SHEET 2 OF 4



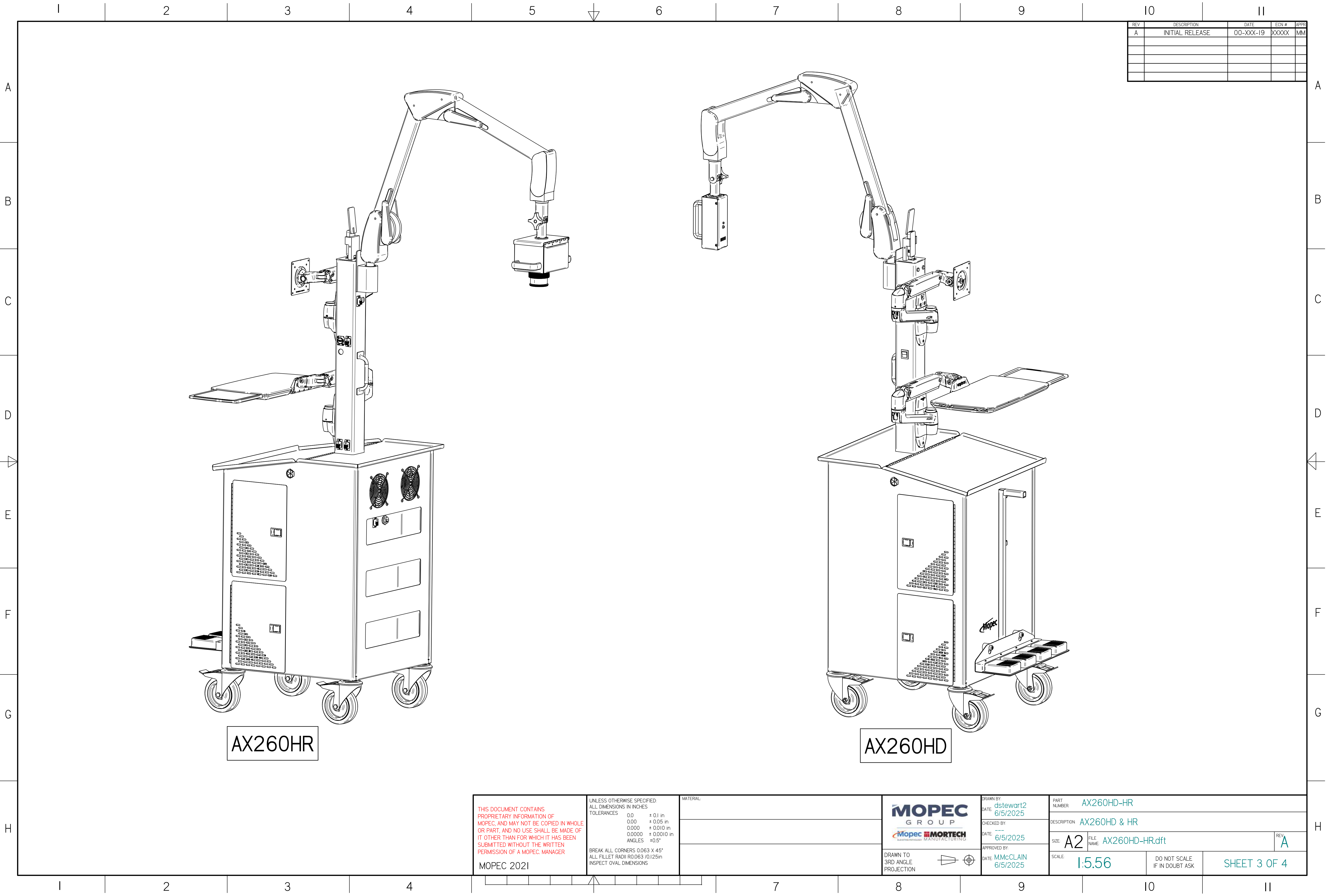


REV	DESCRIPTION	DATE	ECN #	APPR
A	INITIAL RELEASE	00-XXX-19	XXXXX	MM

AX260HR

AX260HD

THIS DOCUMENT CONTAINS PROPRIETARY INFORMATION OF MOPEC, AND MAY NOT BE COPIED IN WHOLE OR PART, AND NO USE SHALL BE MADE OF IT OTHER THAN FOR WHICH IT HAS BEEN SUBMITTED WITHOUT THE WRITTEN PERMISSION OF A MOPEC MANAGER MOPEC 2021	UNLESS OTHERWISE SPECIFIED: ALL DIMENSIONS IN INCHES TOLERANCES 0.0 ± 0.1 in 0.00 ± 0.05 in 0.000 ± 0.010 in 0.0000 ± 0.0010 in ANGLES ±0.5° BREAK ALL CORNERS 0.063 X 45° ALL FILLET RADI R0.063 /0.125in INSPECT OVAL DIMENSIONS	MATERIAL	MOPEC GROUP Mopec MORTECH Manufacturing Technology	DRAWN BY: DATE: dstewart2 6/5/2025 CHECKED BY: DATE: --- 6/5/2025 APPROVED BY: DATE: MMcLAIN 6/5/2025	PART NUMBER AX260HD-HR		DESCRIPTION AX260HD & HR		REV A
					SIZE A2	FILE NAME AX260HD-HR.dft	SCALE 1:5.56	DO NOT SCALE IF IN DOUBT ASK	

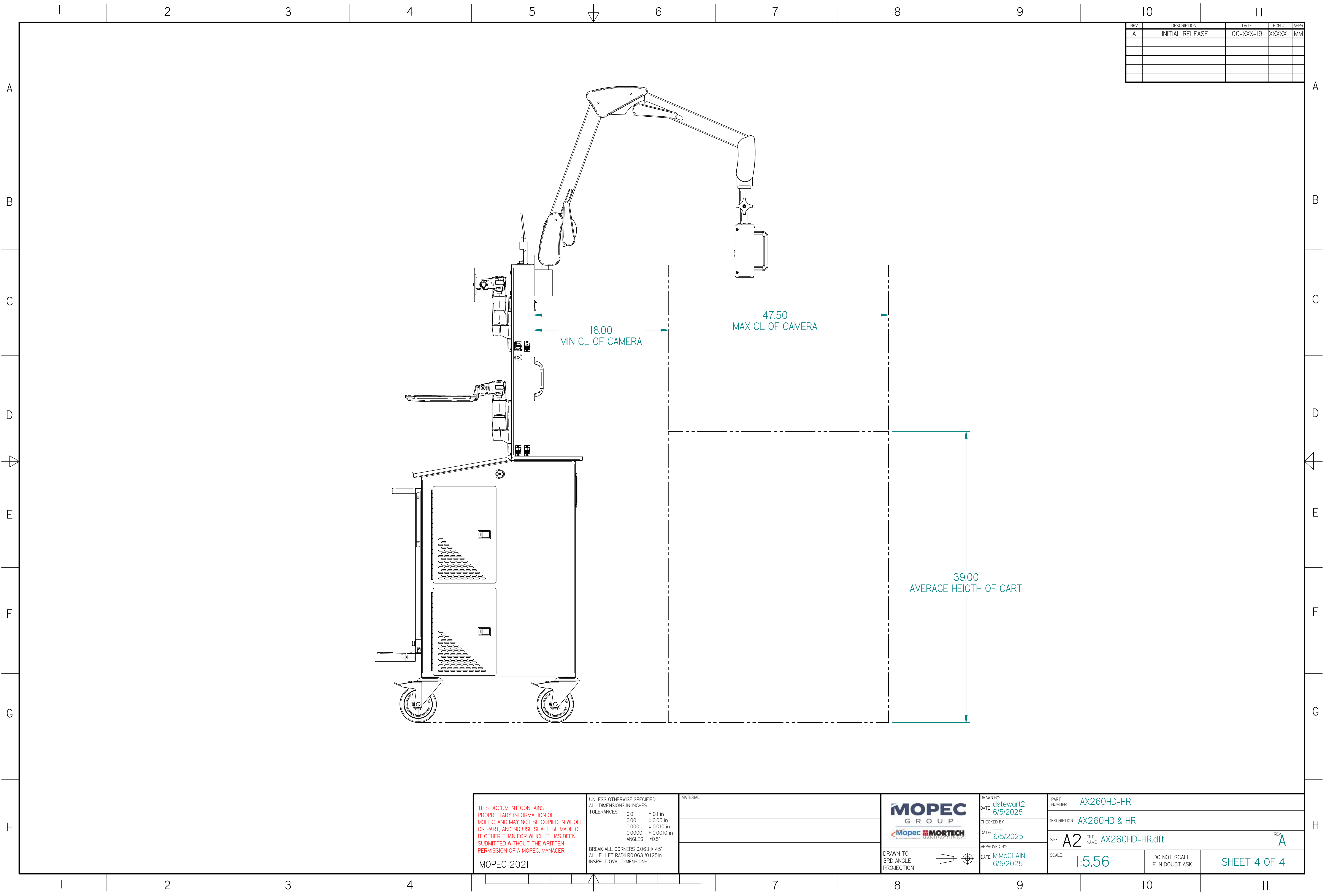

REV	DESCRIPTION	DATE	ECN #	APPR
A	INITIAL RELEASE	00-XXX-19	XXXXX	MM

THIS DOCUMENT CONTAINS PROPRIETARY INFORMATION OF MOPEC, AND MAY NOT BE COPIED IN WHOLE OR PART, AND NO USE SHALL BE MADE OF IT OTHER THAN FOR WHICH IT HAS BEEN SUBMITTED WITHOUT THE WRITTEN PERMISSION OF A MOPEC MANAGER MOPEC 2021	UNLESS OTHERWISE SPECIFIED ALL DIMENSIONS IN INCHES TOLERANCES 0.0 ± 0.1 in 0.00 ± 0.05 in 0.000 ± 0.010 in 0.0000 ± 0.0010 in ANGLES ±0.5° BREAK ALL CORNERS 0.063 X 45° ALL FILLET RADI R0.063 /0.125in INSPECT OVAL DIMENSIONS	MATERIAL	 	DRAWN BY: dstewart2 DATE: 6/5/2025	PART NUMBER AX260HD-HR		
				CHECKED BY: DATE: 6/5/2025	DESCRIPTION AX260HD & HR		REV A
				APPROVED BY: DATE: MMcCLAIN 6/5/2025	SIZE: A2 SCALE 1:5.56	FILE NAME: AX260HD-HR.dft DO NOT SCALE IF IN DOUBT ASK	
DRAWN TO 3RD ANGLE PROJECTION 							

